



ChairPro™ SEAT BELT ALARM

Model #	Fits Wheelchairs	Unit
909370	Universal	Each

Skil-Care's new ChairPro Seat Belt Alarm combines a wheelchair belt with an alarm for double wheelchair safety. The belt attaches to the chair and provides security while the user is seated. When the buckle is opened, an alarm is activated. To assist long-term care facilities maintain a restraint-free environment, Skil-Care has selected a durable buckle that is easily released by most wheelchair users.

PURPOSE


The ChairPro Seat Belt Alarm is designed to alert caregivers when a patient/resident has released the wheelchair safety belt buckle and may leave or fall from the wheelchair.

CAUTIONS

1. The ChairPro Seat Belt Alarm is not a substitute for proper supervision. Fall-risk and wander-risk patients or residents should be monitored regularly.
2. Check battery each time alarm is used. If weak, replace the 9-volt battery.
3. LED should glow when buckle is closed. If LED fails to glow, check battery. Replace if necessary.

INSTALLATION


Alarm Unit

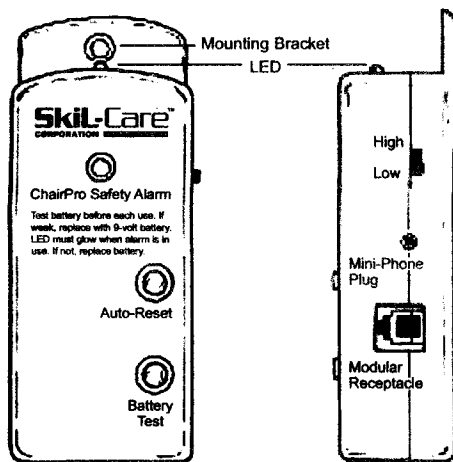


You may choose one of the following two ways to attach the alarm unit to the wheelchair.

Mounting Bracket


1. Remove the second-from-top screw from either the left or right side of the wheelchair backrest.

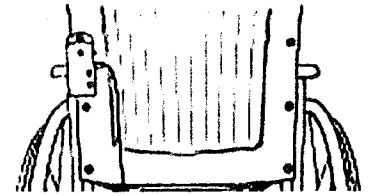


2. Remove grommet from screw, place the screw through hole on alarm unit mounting bracket, and fasten screw to wheelchair backrest. (Illustration A)

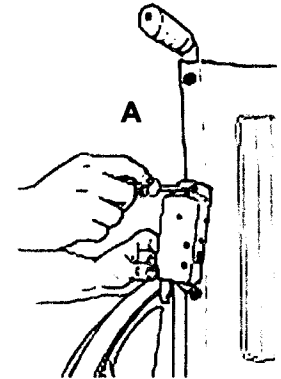

Cord Loop

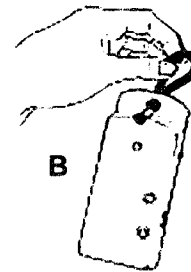


1. Insert enclosed cord loop through bracket hole. (Illustration B)

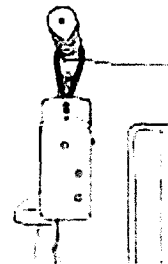

2. Hang from wheelchair push-handle (Illustration C)

Safety Belt


1. Attach the two "spade" connectors on the connector wire to those on the seat belt. (Illustration D)

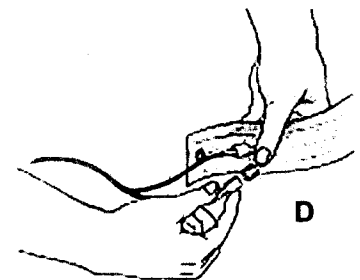


Continued on reverse side.

Safety Belt

2. Remove the rearmost screws from the left and right sides of the wheelchair seat. (Illustration E)

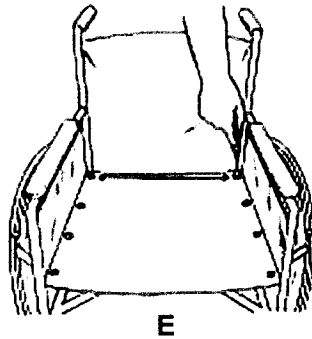

3. Remove the grommets from the screws, place screws through the grommets at the ends of both belt sections, and refasten the screws to the wheelchair seat. (Illustration F)

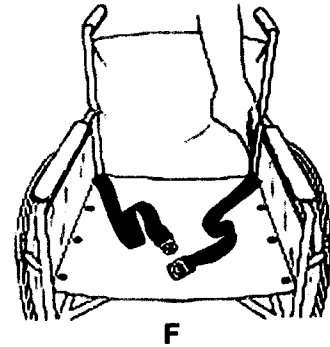

4. Place connector cord through the space between the wheelchair seat and backrest. Make certain that it is away from wheels and other moving parts.

OPERATION


1. Remove battery compartment cover on left side of alarm unit and install 9-volt battery.

2. Press Battery Test button. If alarm doesn't sound, check battery connection and retest. If alarm fails to sound, replace battery.

3. Insert mini-phone jack into Mini-Phone Plug receptacle on right side of alarm. (Illustration G) Alarm will sound. Press Auto-Reset to silence alarm.


4. Before using, test unit by inserting the metal tab into the buckle. Check alarm to make certain that LED is glowing. Then release buckle. Alarm should sound. Press Reset button to silence alarm. If alarm fails to sound, make certain that spade connectors and phone jack are securely in place.

5. Select High or Low alarm volume using switch on right of alarm.

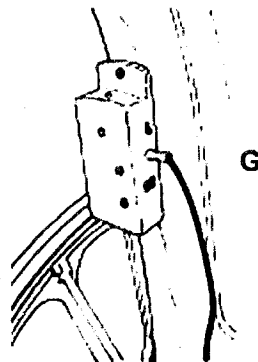

6. Important: If the alarm is activated, it may be silenced by pressing the Auto-Reset button. The alarm is now in stand-by mode. Do not secure buckle when unit is not being used. Doing so will cause the LED to glow and shorten battery life. Alarm will automatically reset itself when the metal tab is inserted into the buckle. There is no On/Off switch.

7. Place belt around user's hips and secure by placing tab into buckle. Adjust belt to proper fit. The belt is released by pressing the "Press" button. (Illustration H)

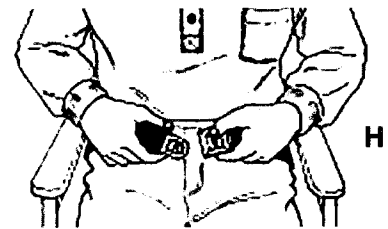

SAFETY REMINDERS

1. Check battery before each use. If alarm is weak or fails to sound, replace 9-volt battery.

2. When unit is in use, check LED regularly. If it is not glowing, replace 9-volt battery.

GUARANTEE

The alarm unit, belt, and buckle are replacement guaranteed against defects in materials and workmanship under conditions of normal use for a period of one year from date of purchase. Please return defective part to dealer from whom the system was purchased. The dealer will send the defective part to Skil-Care and arrange for shipment of a replacement. The guarantee does not cover the battery supplied with the unit.

Skil-CareTM
CORPORATION

Advanced design for nursing and therapy.TM

Alarm unit made in China. Buckle made in Taiwan.

Visit us online
at Skil-Care.com