

PRODUCT SAFETY GUIDELINES

- 1) Please read all instructions before use. Failure to do so may result in injury.
- 2) Make sure all buttons are properly engaged before use.
- 3) Do not exceed maximum weight capacity of 250 lbs. for this item.

ASSEMBLY INSTRUCTIONS

Note: Before assembly please make sure all parts are in box.

- 1) Remove the existing toilet seat and lid from toilet.
- 2) Place the toilet safety frame bracket between the toilet itself and the toilet lid/seat assembly with the rounded part of the bracket facing the back of the toilet.
- 3) Replace the toilet seat/lid and reinsert the mounting bolts and toilet fixture. Replace and tighten nuts.
- 4) Insert the side frames into the mounting bracket and depress detent buttons while sliding both frames into the desired position.
- 5) Adjust foot pieces to height desired and make sure both are at equal heights for stability.
- 6) Before first use make sure all hardware is secure.



CLEANING

Clean item with regular household cleaning products. Do not use abrasive powders or sponges. Clean the grips with mild soap or cleaner, to ensure that plastic does not dissolve and grips do not loosen.

*Please note the leg tips on this product contain dry natural rubber.