Rev. 3 - 10/10_103644

Bauerfeind AG

Weissendorfer Strasse 5
 07937 Zeulenroda-Triebes
 Germany
 Phone +49 (0) 36628-66-40 00
 Fax +49 (0) 36628-66-44 99
 E-mail info@bauerfeind.com

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Stand der Information: Juli 2009

(EN) Dear Customer,

Thank you for choosing a product from Bauerfeind. EpiPoint® is a high-quality medical product.

Please read through these instructions for use carefully and if you have any questions contact your doctor, medical retailer or our customer service department.

EpiPoint® is a product for treating inflammation of the tendons on the outside of the elbow (epicondylitis). It works by applying pressure to the affected forearm muscle. The armband consists of a plastic liner with an integrated steel strap. Rotation of the adjustable pad enables the armband to be worn on both the right and left arms. The pressure of this right/left adjustable pad can be adjusted with the graduated Velcro® fastening.

⚠ EpiPoint® must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum benefits and optimal wearing comfort.

Indications

- Inflammation of the tendons on the outside of the elbow at the lateral epicondyle of the upper arm, e.g. tennis elbow (epicondylitis humeri)

Side effects

Side effects involving the body as a whole, have not been reported to date. Correct fitting is assumed. Any supports and orthoses¹ applied externally to the body can, if tightened excessively, lead to localized pressure or, rarely, constrict the underlying blood vessels or nerves.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.

¹ Orthosis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso

2. Impaired sensation and circulatory disorders of the forearm and the hand.
3. Impaired lymph drainage – including soft tissue swelling of uncertain origin located remotely from the fitted aid.
4. Bottleneck syndromes (irritation due to compression of nerves in the arm).

Fitting EpiPoint®

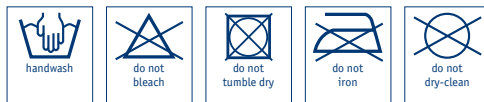
1. Adjust the pad for the left or right forearm: grasp the narrow end of the attached pad, and carefully pull back the Velcro® disk turning the pad to reveal an L (if wearing the product on the left) or a R (if wearing the product on the right). Subsequently affix the pad by pressing the Velcro® disk in place. If the bolt loosens during the positioning of the pad please readjust it through pressure applied from top and bottom.
2. Adjust the Velcro® fastening for the largest circumference and secure. Slip the armband onto the affected forearm so that the rigid portion of the armband always faces outwards.
3. Position the pad approx. 4 - 6 cm below the elbow.
4. To secure the armband in place, lay forearm on a solid support (e.g. table) and tighten the Velcro® fastening, ensuring that the armband is not too tight! An excessively tight fit is indicated by a red warning section on the fastener tab which indicates that the brace should be adjusted by loosening the strap.

Important Information

- EpiPoint® is a prescribed product that should be applied under a physician's guidance. EpiPoint® should only be applied in accordance with the instructions contained in these instructions for use and for the listed areas of application.
- The product is intended to be fitted only once and to one patient.
- No liability is accepted for improper use of the product.
- Changes must not be made to the product. Non-compliance can reduce the performance of the product and thus no liability will be accepted.
- Patients with known contact allergies or who are sensitive to pressure should wear cotton gauze under the EpiPoint®.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- A combination with other products must be discussed beforehand with your medical provider.

- Avoid the use of ointments, lotions or any substances including grease or acids with this product.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.

Product care and washing instructions

EpiPoint® can be washed separately by hand at 30 °C using a mild detergent. To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaints, please contact your retailer or our customer service department.

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