

Posey Abduction Wedges are lightweight, polystyrene filled wedges for FULL LEG and ABOVE THE KNEE abduction. They feature a unique construction of an outer breathable VINYL MESH filled with POLYSTYRENE BEADS. This allows air to flow through the wedge, thereby minimizing the discomfort of conventional foam filled products, which do not breath and can build up heat. The Full Leg Wedge (#6302L) features a zippered closure to allow the addition or removal of beads. 4" wide, soft foam fabric straps with hook closures keep the wedge secured in place between the knees.

POSEY SMALL ABDUCTION KNEE WEDGE

Allows abduction and alignment of the thighs while in a seated position. Supplied with one soft foam fabric strap with hook fasteners.

REF 6302S

POSEY FULL LEG ABDUCTION WEDGE

Supplied with two soft foam fabric straps with hook fasteners.

REF 6302L



6302S

APPLICATION INSTRUCTIONS:

1. Position the patient as desired.
2. Place the wedge on the patient ensuring that there is no discomfort.
3. Position the Abduction Wedge in place with the soft foam strap(s) and secure with the hook closures sewn into the strap.
4. Monitor the patient at least every two hours for skin irritation and/or pressure points.
5. Reposition the patient as necessary.

WASHING INSTRUCTIONS:

Machine wash in warm temperature. Air dry preferred.



6302L

Description	Posey Item #	Dimensions			Replaces MEDCO Item #
		Length	Thickness	Width	
Knee Abduction Wedge	#6302S	8"	5"	7" tapering to 5"	1AW-S
		20 cm	13 cm	18 cm tapering to 13 cm	
Full Leg Abduction Wedge	#6302L	22"	5"	15" tapering to 7"	1AW-L
		56 cm	13 cm	38 cm tapering to 18 cm	
Bead Refill Kit (1 lb.)	#8124	N/A	N/A	N/A	3PK

Materials used meet the requirements of FMVSS-302 and CA #117 Section E for Flammability.