ThermalSoft® Gel

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Email: mail@mettlerelectronics.com

Web site: www.mettlerelectronics.com

Copyright © 2001 by Mettler Electronics Corp.—Anaheim, CA. All rights reserved.

Instructions for Use

For Cold Therapy

1. Chill at least two (2) hours in the freezer. Do not chill colder than 0° F (-17° C).
2. Remove the ThermalSoft Gel Pack from the freezer and wrap in a single thickness of towel.
3. Apply to the body for no more than 30 minutes.

Precautions for Cold Therapy

Warning: Do not place the cold ThermalSoft Gel pack directly on the skin as frostbite can result.

Caution: Do not apply to insensitve skin or in areas of poor circulation or nerve damage.

Do not sit, lie or sleep on the ThermalSoft Gel Pack.

Monitor treatment area to insure that skin is not becoming blanched or irritated.

For Heat Therapy

1. Heat the ThermalSoft Gel Pack in the microwave for 2-4 minutes. *See table below for additional guidelines on heating.* Wrap free ends of the neck/shoulder and extra large pack under the pack to assist with even heating. *Heating time varies due to the size of the pack and the power of your microwave oven.*

Pack	Initial time	Turn over	Second time
Standard	4 minutes	Yes	2 minutes
Half	3 minutes	No	
Cerv/Sh	4 minutes	Yes	2 minutes
X-Lge	8 minutes	Yes	2 minutes

2. Carefully remove the pack from the microwave and place on several layers of toweling. *Do not place heated ThermalSoft pack directly on the skin as severe burns can result.*
3. As the pack cools, you may remove layers of toweling. Apply to the body for no more than 20-25 minutes.

Precautions for Heat Therapy

Warning: Do not place the heated ThermalSoft pack directly on the skin as severe burns can result.

Caution: Do not apply to insensitve skin or in areas of poor circulation or nerve damage.

Do not sit, lie or sleep on the ThermalSoft Gel Pack.

Check to make sure the pack does not feel too hot before applying to injured area.

Monitor treatment area to insure that skin is not becoming irritated.

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