

Floor Cushions

Simple precautions can help create a safer environment. The Posey Floor Cushions help to minimize injuries from a fall.

Maximum Protection

Cushions absorb up to 85% of impact forces.* The 1" (3 cm) EVA foam creates a low profile that is easy for patients to step on and off. The non-skid bottom helps prevent slipping when stepping onto the Cushion. Meets FMVSS302 flammability standards. To cover a larger area, simply connect the straight sides of two Cushions using the hook-and-loop strips provided.

- REF 6020** Standard Floor Cushion: 67"L x 26"W x 1"H
(170 cm x 66 cm x 3 cm)
- REF 6021** Two-Tone Floor Cushion: 67"L x 26"W x 1"H
(170 cm x 66 cm x 3 cm)
- REF 6023** Beveled Floor Cushion: 70"L x 38"W x 1"H
(178 cm x 97 cm x 3 cm)
- REF 6025** Beveled Floor Cushion, Wide: 70"L x 44"W x 1"H
(178 cm x 112 cm x 3 cm)
- REF 6026** Beveled Floor Cushion, Narrow: 70"L x 29"W x 1"H
(178 cm x 74 cm x 3 cm)
- REF 6027** Beveled Floor Cushion, Premium: 70"L x 38"W x 1"H
(178 cm x 97 cm x 3 cm)
- REF 6027R** Beveled Floor Cushion, Posey Glow (glow strip on the edge):
70"L x 38"W x 1"H (178 cm x 97 cm x 3 cm)

Cushions with beveled edges easily accommodate wheelchairs, walkers, and table trays. The Two-Tone Cushion is easier to see in low light conditions, or for a sight impaired patient.

Use Instructions:

1. Undo the hook-and-loop closure and unfold the Floor Cushion.
2. Make sure the "This Side Down" label is facing the floor.
3. Position the Cushion lengthwise, with the straight (non-beveled) edge flush to the side of the resident's bed.
4. To connect two Cushions, use "hook" and "loop" strips. Remove the paper backing and apply strips to the straight edges of the two Cushions.

⚠ WARNING Inform resident and staff to take care when stepping on or off of the Cushion to avoid tripping.

Follow your facility's procedures for cleaning spills in patient care areas.

DO NOT place Floor Cushions on or around damp or wet areas. Clean spills on, around or under Floor Cushions. Liquids under Floor Cushions may cause a slip and fall accident.

⚠ CAUTION When used with a Low Bed, be sure to leave enough room between the bed and the Cushion. If the Cushion is too close when the handrail is rotated, it could tear the Cushion cover.

⚠ CAUTION Posey Glow strips will fade after two hours unless recharged by light exposure for approximately twenty minutes.

Easy to Clean and Store

Vinyl cover easily wipes clean with liquid disinfectant and a soft cloth. When not in use, fold the Cushion in thirds for easy, compact storage. When folded, Cushions are only 3" (8 cm) thick. Use the hook-and-loop closure as a carrying handle.




Disposal

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

* All models tested according to ASTM protocols for shock absorbency.



6020

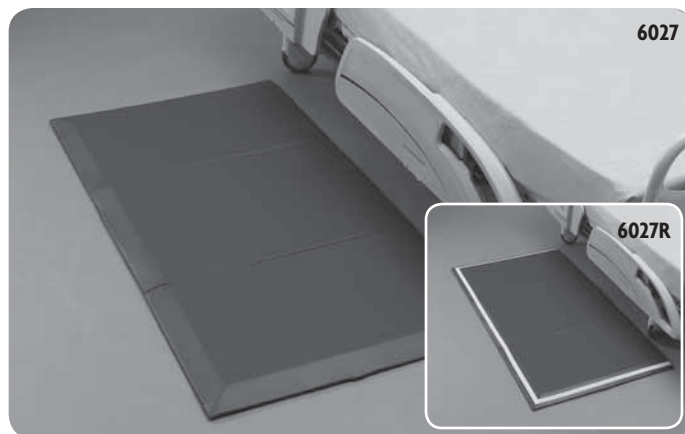

6027

6027R



Connect two Floor Cushions to create a wider area of coverage

NOTE: Due to the random possibility of falls, the Posey Company makes no guarantee, express or implied, that the user is protected from hip trauma.

Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.