## Rolyan® Climbing Board:


A3591

## Rolyan® Climbing Accessory Kit:


A35910

A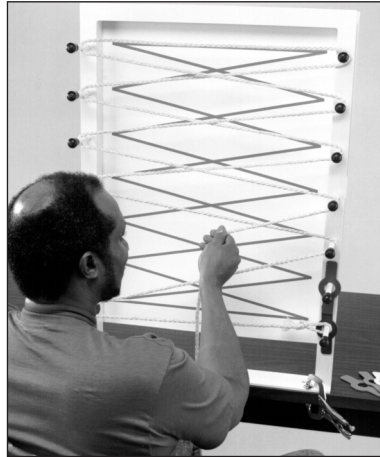

B

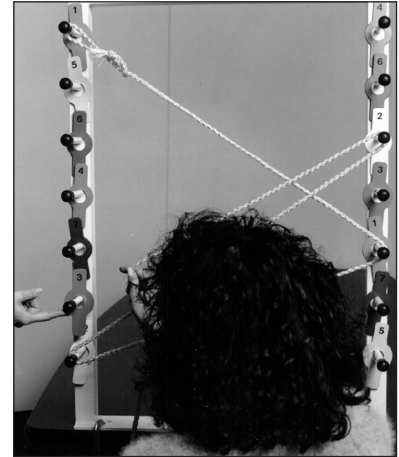C

### INDICATIONS

- To increase upper extremity range of motion, strength, and coordination
- To improve eye/hand coordination
- To improve the ability to follow a sequence and simple directions
- To improve balance and trunk control
- To improve the ability to compensate for hemianopsia and overcome unilateral neglect
- To improve the ability to cross the midline

### PRECAUTIONS

- This equipment should be used only with the instruction and supervision of a healthcare professional.
- If unusual discomfort occurs, use should be discontinued.
- Be sure that the frame is attached securely to the wall or table top.

### CARE AND CLEANING

The metal and plastic parts can be cleaned with a hard-surface disinfectant.

### PROCEDURE

#### Preparing climbing board and accessory kit for use

1. **Rolyan Climbing Board:** Mount frame to a wall or a table top by one of the following methods:
  - For attachment to a wall, drill holes in the sides of the climbing board frame and attach it to an appropriate apparatus on the wall.
  - For permanent attachment to a table top, drill holes in the table top to line up with the holes in the base. Attach with appropriate bolts.
  - For temporary attachment, use vise grips or C-clamps as shown in B.

2. **Rolyan Climbing Board Accessory Kit:** Self-adhesive hook is attached to the loop on the "X" pattern. Attach it to the frame of the climbing board by removing the paper backing and pressing the pattern (with the hook in place) onto the back of the frame of the climbing board. Carefully remove the pattern board and press the hook pieces firmly into place.

#### Using the climbing board and accessory kit

3. Suggested use of the Rolyan Climbing Board is shown in A.
4. Suggested uses of the Rolyan Climbing Board Accessory Kit include the following:
  - Ask the patient to string rope across the climbing board frame following the pattern on a board (B).
  - Attach the adhesive numbers to the colored tabs, using a single number per color. Place the tabs over the extension rods of the frame. Ask the patient to follow a sequence (C).

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---

**Distributed by:**  
**Performance Health Supply Inc.**  
28100 Torch Parkway, Ste. 700  
Warrenville, IL 60555, USA  
Phone: 800-323-5547  
Fax: 800-547-4333

**Manufacturer:**  
**Performance Health Supply Inc.**  
W68 N158 Evergreen Blvd.  
Cedarburg, WI 53012, USA  
Phone: 800-228-3693

**Distributed by:**  
**Performance Health Canada, Inc.**  
6675 Millcreek Drive, Unit 3  
Mississauga, ON L5N 5M4  
Phone: 800-665-9200  
Fax: 877-566-8611

**Authorized European  
Representative**  
**Performance Health  
International, LTD.**  
Nunn Brook Road  
Huthwaite, Sutton-in-Ashfield  
Nottinghamshire  
NG17 2HU, UK  
**International**  
Phone: +44 (0) 1623 448 706  
Fax: +44 (0) 1623 448 784