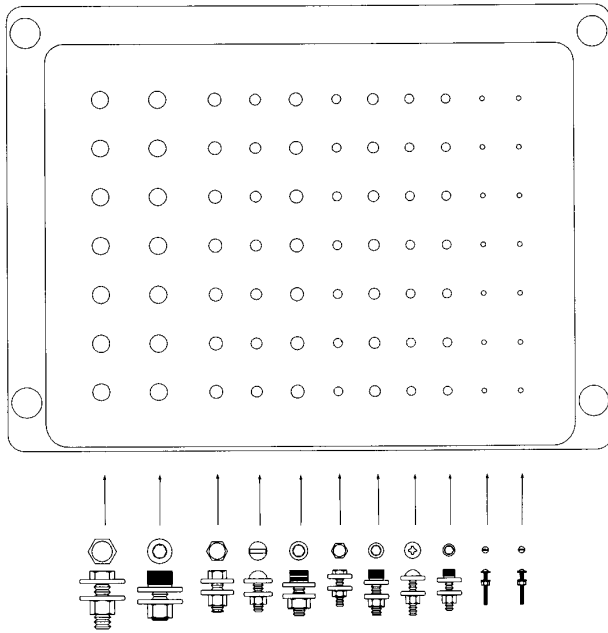
A

Nuts, bolts, washers, and socket screws in line with corresponding rows.

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INDICATIONS

- To increase upper extremity range of motion, strength, and coordination
- To increase fine motor prehension
- To reduce hypersensitivity in the finger tips
- To improve eye/hand coordination
- To improve the ability to follow a sequence and simple directions
- To improve balance and trunk control
- To improve the ability to compensate for hemianopsia and overcome unilateral neglect
- To improve the ability to cross the midline

PRECAUTIONS

- **This equipment should be used only with the instruction and supervision of a healthcare professional.**

- **If unusual discomfort occurs, use should be discontinued.**

CARE AND CLEANING

All parts can be cleaned with a hard-surface disinfectant.

PROCEDURE

Assemble the Rolyan Work Bench according to diagram **A**.

Activities can be performed with or without the hand tools. The work bench can be positioned to provide work simulation activities while the patient is sitting, standing, or lying. The cutout area allows access to the bottom of the work bench to provide bilateral activity in a confined work space.

An assembled Rolyan Work Bench is shown in **B**.

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Patterson Medical, Ltd.