INDICATIONS

- For use in combination with an external fixation system to provide dorsiflexion assist for tibial fractures and help prevent foot drop
- May be used for plantar flexion exercise to maintain ankle ROM and strength

CONTRAINDICATIONS

- Not for fixed bony contractures
- Not for use over open wounds and infected areas

PRECAUTIONS

- **This product is not to be used during ambulation.**
- **This product is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners,**

care providers involved in the patient's care, and the patient.

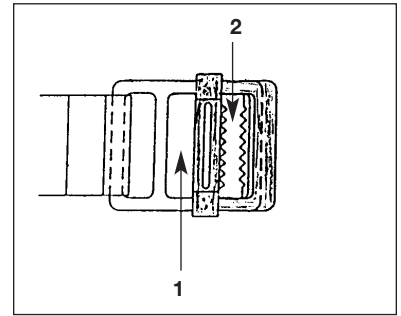
- **If unusual swelling, discomfort, or skin discoloration or breakdown occurs, use should be discontinued and a healthcare professional consulted.**
- **This product contains natural rubber latex, which may cause allergic reactions.**
- **Be careful that the straps are not so tight that they interfere with circulation.**

CARE AND CLEANING


- The open cell foam insert can be wiped clean. The plastic splint and straps can be washed with soap and warm water. All parts should be allowed to dry thoroughly before reapplication.
- Replacement kit, which includes foam, elastic straps, and strap pad, is available.

INSTRUCTIONS FOR USE

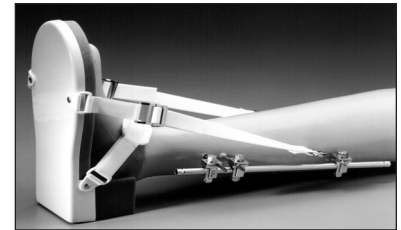1. With all straps open, place heel in contoured heel space.
2. Fasten D-ring strap across foot.
3. Attach spring snaps that are on loose straps to fixator frame.
4. Thread loose end of each strap into a buckle on the support as follows (A):
 - Thread it through opening 1 from back to front.
 - Insert it between teeth of opening 2 from front to back.
 - Pull free end of elastic to adjust length.
 - Pull elastic that is attached to spring snap to tighten strap in buckle.
 - Adjust foot positioning by lengthening or shortening straps.
5. A properly applied Rolyan External Fixator Foot Support is shown in B.

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