# Rolyan® Biodynamic® Flexion/Extension System:

A572-001 and A572-002

Figures referred to in the instructions are on page 7.

## INDICATIONS

- To provide dynamic traction for flexion or extension
- To provide static progressive flexion or extension
- Can be used following tendon repairs and grafts, crush injuries, nerve repairs, and burns
- Can be used to help reduce PIP and IP joint stiffness

## CONTRAINDICATIONS

- Not for use in the presence of severe edema, decreased circulation, and fixed bony contractures
- Not for use during the acute phase of fractures

## PRECAUTIONS

- **This product is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps and finger wraps are not so tight that circulation is impaired.**
- **The patient should never make any adjustment to the splint or strapping system. If adjustments are necessary, the product should be removed and a healthcare professional consulted.**

## CARE AND CLEANING

Straps can be washed with soap and water. They should be allowed to dry thoroughly before reapplication.

## INSTRUCTIONS FOR USE

Directions for using the gauntlet from the Rolyan Finger Rehab System instead of a splint base begin with instruction 20.

Directions for increasing extension begin with instruction 24.

### Using system to increase flexion

1. Fabricate splint base as appropriate for the diagnosis.

#### **Attaching line guides**

To encourage composite flexion, line guides on palmar bar of splint are recommended. Attach one line guide for each involved finger.

2. Hold fingers in passive flexion. Allow room between fingers for finger wraps. Note that

fingers should converge toward scaphoid when splint and strapping system are applied. Mark mid point of each digit on palmar bar of splint (A).

3. Remove splint from patient.
4. If splint material has protective coating, remove it with solvent or by scraping with sharp instrument.
5. Holding line guide with needle-nose pliers, quickly heat flat portion of line guide with heat gun.
6. Position heated line guide over mark on splint so eyelet is approximately 1/4" (6.4 mm) from distal edge (B). Gently mold line guide into splint. Repeat for necessary line guides.
7. Heat **Polyform®** attachment piece in heat pan of water heated to 150° to 160°F (65°–70°C) for approximately one minute. Remove from water, pat dry, and slide hole in attachment piece over eyelet portion of line guide (C).

#### **Applying straps**

8. Slide forearm pad onto forearm strap so eyelets in strap are toward the outside when strap is applied (D).
9. With splint on patient, position forearm strap near proximal edge of splint so eyelets in strap are centered on volar surface of forearm. Attach self-adhesive hook pieces to splint to secure strap.
10. Position wrist strap so line guide is toward the outside and centered on volar surface of wrist. Attach self-adhesive hook pieces to splint to secure strap.

#### **Attaching finger wrap(s) and monofilament(s) to splint**

For each involved finger, a distal and proximal finger wrap attached to monofilament is necessary.

11. If line guides on palmar bar of splint are used, bring together both ends of monofilament and feed them through corresponding distal line guide (E).
12. Feed monofilament through line guide on wrist strap (E).
13. Attach traction device for each involved finger through large hole in tension-adjustable piece and eyelet in forearm strap that corresponds to finger.
14. Attach monofilament to tension-adjustable piece by sliding both monofilament ends down through distal hole and back up through proximal hole (F).

#### **Applying finger wraps**

15. With splint on patient, wrap proximal (larger) finger wrap around proximal phalanx, being sure the four eyelets are on volar surface. Secure closure.
16. Place skid-resistant material of distal (smaller)

finger wrap over fingernail and wrap around finger, securing closure. The two eyelets should be on volar surface of finger.

#### **Checking splint**

17. Check for proper traction force and adjust if necessary by changing traction device or increasing or decreasing length of monofilament.
18. Check all straps. Trim as necessary. Be careful not to trim close to eyelets and embedded line guide.
19. A completed system on a dorsal-based radial bar splint is shown in G.

### Using system to increase flexion using gauntlet from Rolyan Finger Rehab System

20. Apply finger wraps to fingers (see instructions 15 and 16). Apply gauntlet as shown in H.
21. Thread loose ends of monofilament through eyelet on tension adjustment strip and tie them.
22. For static traction, attach hook material on tension adjustment strip to volar surface of gauntlet (H).
23. For dynamic traction, attach rubber band to tension adjustment strip. Tie monofilament through loop of rubber band.

### Using system to increase extension

24. Fabricate splint base as appropriate for the diagnosis.
25. Punch hole in distal portion of splint base corresponding to each affected joint (I).
26. Separate the two finger wraps by removing monofilament from proximal finger wrap. Thread loose piece of monofilament through proximal finger wrap.
27. Attach pieces of loop material to dorsal surface of splint (I).
28. With splint on patient, wrap proximal (larger) finger wrap around proximal phalanx, being sure the four eyelets are on volar surface. Secure closure.
29. Place skid-resistant material of distal (smaller) finger wrap over fingernail and wrap around finger, securing closure. The two eyelets should be on volar surface of finger.
30. For static traction, thread loose ends of monofilament through eyelet on tension adjustment strip and tie them. Attach hook material on strip to dorsal surface of splint (I and J).
31. For dynamic traction, attach rubber band to tension adjustment strip. Tie monofilament through loop of rubber band.

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**Patterson Medical, Ltd.**