

Rolyan® Introductory Splinting Kit

A783-101

The Rolyan Introductory Splinting Kit is designed to be used with the book, *Introduction to Splinting: A Critical-Thinking and Problem-Solving Approach* (Product No. 5628-79).

The splinting kit contains splinting materials and other supplies necessary to make the splints described in laboratory exercises in the book. The table below lists the splinting material that is recommended for each splint. For directions about softening and working with each material, refer to the instruction sheet for the splinting materials.

The 2" by 2" (5 x 5cm) pieces of splinting material in the kit are to be used with Exercise 1-1 in *Introduction to Splinting*. Soften each sample, one at a time, and answer the questions about the material's properties and characteristics.

In addition to the splinting materials, the kit contains the following splinting products:

- Strapping materials, which are to be used wherever straps are required (note that the 2"-wide [5cm] strapping material may need to be cut lengthwise to provide a narrower strap)

- Outrigger wire, finger loops, and rubber bands for the dynamic extension (radial nerve) splint

- A 6" (15cm) goniometer, to help ensure that joints are immobilized at an appropriate angle

Additional items are needed to make the splints, such as a heat pan or Hydrocollator unit, a heat gun, scissors, and a solvent. Sammons Preston offers a full range of splinting supplies and accessories in its rehabilitation products catalog.

Type of splint	Recommended splinting material	Exercise in <i>Introduction to Splinting</i>
Wrist cock-up	Ezeform®, 3/32," (2.4mm) thick, 9" x 12" (23 x 30 cm)	Exercise 7-3
Resting hand	Ezeform®, 1/8," (3.2mm) thick, 12" x 18" (30 x 46 cm)	Exercise 9-3
Thumb spica	Aquaplast® ProDrape®-T, 1/8," (3.2mm) thick, 9" x 12" (23 x 30 cm)	Exercise 8-4
Dynamic extension (radial nerve)	Synergy®, 1/8," (3.2mm) thick, 9" x 12" (23 x 30 cm)	Exercise 11-1
Ulnar nerve	TailorSplint™, 1/8," (3.2mm) thick, 9" x 12" (23 x 30 cm)	Exercise 13-2



Rev. 5, Effective: January 2015
Patterson Medical Products, Inc.
9104.605

Distributed by:
Patterson Medical
28100 Torch Parkway, Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Direct Phone: 630-393-6000
Fax: 800-547-4333
Direct Fax: 630-393-7600

Patterson Medical
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012, USA
Phone: 800-323-5547
Fax: 800-547-4333

Patterson Medical Canada
6675 Millcreek Drive, Unit 3
Mississauga, ON L5N 5M4
Phone: 800-665-9200
Direct Phone: 905-858-6000
Direct Fax: 905-858-6001
Toll-Free Fax: 877-566-8611

Authorized European Representative
Patterson Medical, Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784

Rolyan® Introductory Splinting Kit

A783-101

The Rolyan Introductory Splinting Kit is designed to be used with the book, *Introduction to Splinting: A Critical-Thinking and Problem-Solving Approach* (Product No. 5628-79).

The splinting kit contains splinting materials and other supplies necessary to make the splints described in laboratory exercises in the book. The table below lists the splinting material that is recommended for each splint. For directions about softening and working with each material, refer to the instruction sheet for the splinting materials.

The 2" by 2" (5 x 5cm) pieces of splinting material in the kit are to be used with Exercise 1-1 in *Introduction to Splinting*. Soften each sample, one at a time, and answer the questions about the material's properties and characteristics.

In addition to the splinting materials, the kit contains the following splinting products:

- Strapping materials, which are to be used wherever straps are required (note that the 2"-wide [5cm] strapping material may need to be cut lengthwise to provide a narrower strap)
- Outrigger wire, finger loops, and rubber

bands for the dynamic extension (radial nerve) splint

- A 6" (15cm) goniometer, to help ensure that joints are immobilized at an appropriate angle

Additional items are needed to make the splints, such as a heat pan or Hydrocollator unit, a heat gun, scissors, and a solvent. Sammons Preston offers a full range of splinting supplies and accessories in its rehabilitation products catalog.

Type of splint	Recommended splinting material	Exercise in <i>Introduction to Splinting</i>
Wrist cock-up	Ezeform®, 3/32," (2.4mm) thick, 9" x 12" (23 x 30 cm)	Exercise 7-3
Resting hand	Ezeform®, 1/8," (3.2mm) thick, 12" x 18" (30 x 46 cm)	Exercise 9-3
Thumb spica	Aquaplast® ProDrape®-T, 1/8," (3.2mm) thick, 9" x 12" (23 x 30 cm)	Exercise 8-4
Dynamic extension (radial nerve)	Synergy®, 1/8," (3.2mm) thick, 9" x 12" (23 x 30 cm)	Exercise 11-1
Ulnar nerve	TailorSplint™, 1/8," (3.2mm) thick, 9" x 12" (23 x 30 cm)	Exercise 13-2



Rev. 5, Effective: January 2015
Patterson Medical Products, Inc.
9104.605

Distributed by:
Patterson Medical
28100 Torch Parkway, Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Direct Phone: 630-393-6000
Fax: 800-547-4333
Direct Fax: 630-393-7600

Patterson Medical
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012, USA
Phone: 800-323-5547
Fax: 800-547-4333

Patterson Medical Canada
6675 Millcreek Drive, Unit 3
Mississauga, ON L5N 5M4
Phone: 800-665-9200
Direct Phone: 905-858-6000
Direct Fax: 905-858-6001
Toll-Free Fax: 877-566-8611

Authorized European Representative
Patterson Medical, Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784