



Myossage[®] Lotion Warmer




Instruction Sheet

Models: TM-1

Note: Stainless steel screws are provided with your Myossage Lotion Warmer - for use in mounting the unit on any wood treatment table.

1. Remove the door-bottle rack assembly by lifting gently upward by the black plastic knob. This disengages the two projecting rods on the bottom of the bottle rack and permits the bottom of the assembly to be pulled outward from the front opening.
2. Remove the assembly by lowering downward out of the opening. This provides accessibility to either of the two side holes for mounting to the leg of the table.
NOTE: The stainless steel angle attached to the top of the Lotion Warmer has slotted holes to permit adjustments in and out to suit specific requirements.
3. Hold the Lotion Warmer in place and mark (by nail or pencil) the locations of the holes as shown. (Holes A for the top angle; B for the side hole). These holes can be started by means of a drill with a small bit or more simply with a small nail and hammer.
4. When the holes are located and started, the Lotion Warmer can be held in place and the screws tightened. Start with the two screws in the top angle.
5. Reinstall the door-bottle rack assembly by inserting the assembly up into the front opening. This will enable the projecting rods to go up over the bottom lip of the front opening. Tilt the assembly backward into place as shown.

NOTE: The Lotion Warmer operates at temperatures near or slightly above normal body temperature. Actual temperature of lotion inside bottles may vary. The Myossage Lotion Warmer is heated by a specially designed low-wattage heating element. It is advisable to leave the Lotion Warmer plugged in and operating continually. The switch on the front of the Lotion Warmer contains a pilot light to indicate that the Lotion Warmer is on and operating. Bottles should be shaken before use in order to evenly distribute heat.

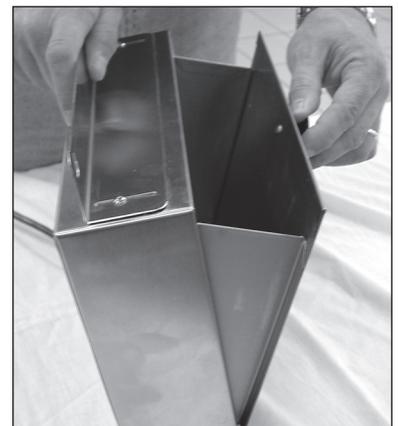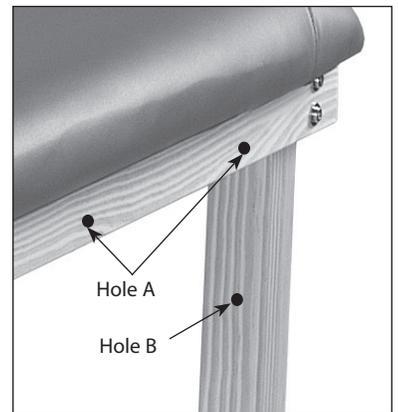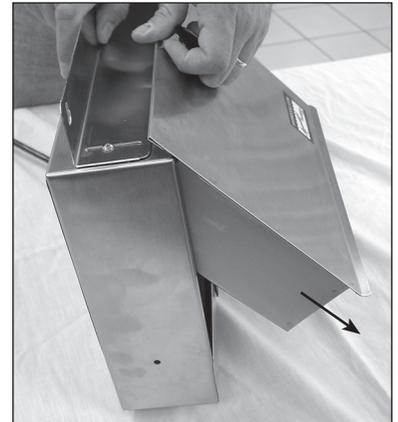

ISO 13485 Certified

DJO, LLC
1430 Decision St
Vista, CA 92081 USA
Phone: 1-800-592-7329 USA
Phone: 1-423-870-2281 or
1-317-406-2250
Fax: 1-317-406-2014
chattgroup.com

© 2009 DJO, LLC

21986_F



2 1 9 8 6