


## CALF PAD COVER

Reorder #	Unit
703068	Pair

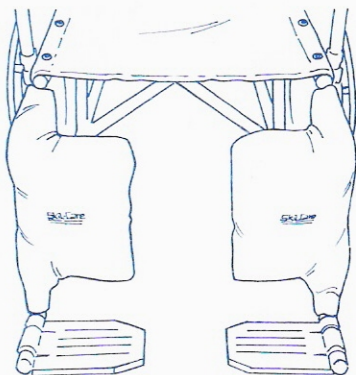

### Purpose

Skil-Care's Calf Pad Cover is designed to reduce pressure on calves and protect the lower leg against skin tears and abrasions caused by contact with the wheelchair leg rest support bars.


### Application

Note: The pads are sold in pairs and are labeled "left" and "right".


1. Slide wheelchair calf pad into pad cover sleeve and secure Velcro® backstrap. (Illustration A)

2. Wrap side of calf pad cover around the top and bottom of wheelchair foot support bar and secure Velcro® closures. (Illustration B)

A

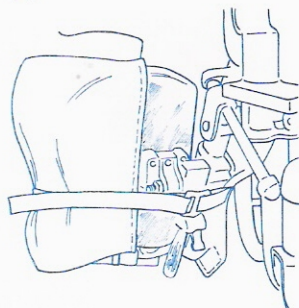

B

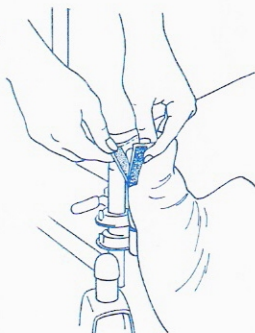

### Easy Care

The Calf Pad Covers are covered with Skil-Care's Low-Shear fabric that wipes clean with a mild spray-type cleanser. The cover may be disinfected with a non-chlorine disinfectant. Do not launder.