

# THRESHOLD RAMP

## FOR WHEELCHAIRS AND SCOOTERS

Available in 1-, 1½-, 2-, 3-, 4-, 5-, & 6-inch Heights

# EZ-ACCESS®

## INSTRUCTIONS FOR USE

### FOR YOUR SAFETY:

- ⚠ Maximum weight limit is 600 pounds.
- ⚠ Read and follow all instructions before use.
- ⚠ Do not use if ramp is damaged or unstable.
- ⚠ It is important that you refer to your equipment's (wheelchair or scooter) User Guide for the proper degree of incline/decline and chair direction before attempting ramp use. **Never exceed its recommendations.**
- ⚠ Use ramp only with a qualified helper and always use your lap belt.

### PACKAGE CONTENTS:

#### THRESH 1 (FIGURE 1)

(A) foot, (C) flap, ¼" screws

#### THRESH 1.5 (FIGURE 1)

(A) foot, (B) 1½" extension, (C) flap, ¼" screws

#### THRESH 2 (FIGURE 2)

(A) foot, (B) 2" extension, (D) flap, ¼" screws

#### THRESH 3 (FIGURE 2)

(A) foot, (B) 2" extension, (C) 3" extension, (D) flap, ¼" screws

#### THRESH 4 (DIAGRAM 2)

(A) foot, (E) flap, ¼" screws


#### THRESH 5 (DIAGRAM 2)

(A) foot, (B) 1½" extension, (E) flap, ¼" screws


#### THRESH 6 (DIAGRAM 2)

(A) foot, (C) 2" extension, (E) flap, ¼" screws


### FIGURE 1

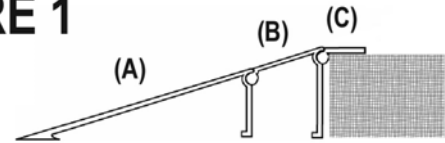

### FIGURE 2

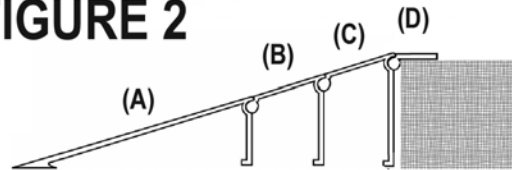

### FIGURE 3

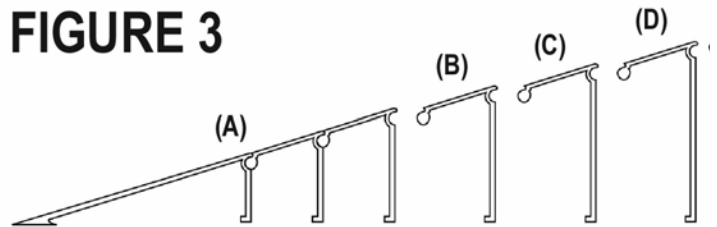

1. EZ-ACCESS, a division of Homecare Products, Inc., herein referred to as Manufacturer, warrants that the equipment is free from defects in material and workmanship under normal use and service. This limited warranty is effective for a period of 3-years from the date of original retail purchase (**original dated purchase receipt is required**) and applies to metal structure only (excludes powder coated finish and applied non-skid tread). The provisions of this warranty shall not apply to any equipment which has been subject to misuse, vandalism, negligence, alteration, accident, or improper installation, or which has been repaired outside Manufacturer's place of business in any way as, in the reasonable judgment of Manufacturer, to adversely affect its performance and reliability, nor to normal deterioration due to wear, tear and exposure, corrosion, damage caused by rain, fire, earthquake or other natural causes or acts of nature.
2. Manufacturer's obligation under this warranty is limited to repairing or replacing, at its option, any equipment which is returned to Manufacturer's place of business with return authorization number (SEE #4 FOR DETAILS) and which, upon examination, shall disclose to Manufacturer reasonable satisfaction to have been thus defective. Manufacturer will make the repair or replacement of defective parts under this warranty without charge for parts or labor.
3. If Manufacturer finds that any equipment has not provided the warranted service life, Manufacturer will, as its sole obligation and the user's sole remedy for breach of this warranty, repair or replace the unit, at its option, for a charge prorated on the following basis. The user will be allowed a credit against the list price of equivalent equipment at the time of return to Manufacturer, in proportion to the percentage of the warranted service life remaining at the time of return to Manufacturer. Purchaser will be responsible to pay the balance of the list price of equivalent equipment; and Manufacturer reserves the right to require payment prior to delivery of the repaired/replacement equipment. Manufacturer's whole warranty responsibilities are summarized below (return shipping/delivery charges to be paid by the Purchaser):
  - YEARS 0-1: unit repaired or replaced free of charge.
  - YEARS 1-2: 50% \$ credit (current list price multiplied by credit percentage. In calculating the available credit, the remaining warranted service life will be rounded up or down to the nearest whole year).
  - YEARS 2-3: 25% \$ credit (current list price multiplied by credit percentage. In calculating the available credit, the remaining warranted service life will be rounded up or down to the nearest whole year).
4. The attached registration card must be mailed within 10-days after purchase to the Manufacturer in order to validate the warranty. If Purchaser discovers a defect, he/she must call Manufacturer at 253-249-1101 and request an RA number to return, at his/her own expense, the merchandise to Manufacturer. Additionally, digital images of merchandise (close up of defect, plus photo of overall product) must be submitted (an email address will be provided at the time an authorization number is given). The duration of any implied warranty of merchantability or fitness for a particular purpose shall be limited in all respects to the duration of the limited warranty and the warranty described above shall be in lieu of any other express warranty. Some states do not allow limitation on how long an implied warranty will last so the above limitation may not apply to you. Manufacturer neither assumes nor authorizes any other person to assume for it any other liability in connection with Manufacturer.
5. The buyer's sole and exclusive remedy against the Manufacturer shall be for the repair or replacement of defective merchandise as provided above. No other remedy, including but not limited to, incidental or consequential damages for lost profits, lost sales, injury to property, or any other incidental or consequential loss shall be available to him/her. Because some states do not allow the exclusion or limitation of incidental or consequential damages, the above limitation or exclusion may not apply to you. Before using, user shall determine the suitability of the product for its intended use and user assumes all risk and liability whatsoever in connection therewith.
6. This warranty gives you specific legal rights. You may also have other rights, which vary from state-to-state.



# THRESHOLD RAMP

## FOR WHEELCHAIRS AND SCOOTERS

Available in 1-, 1½-, 2-, & 3-inch Heights



# INSTRUCTIONS FOR USE

### SINGLE RAMP APPLICATION STANDARD THRESHOLD & LANDING



- Slide each section together and insert screw as shown in (DIAGRAM 1). Assemble the ramp by connecting the individual sections as shown in (DIAGRAM 2 or 3).
- Place the ramp against the threshold or landing (DIAGRAM 2). The adjustable flap should rest on your threshold or landing to provide a smooth surface for your wheels. If the adjustable flap is not used, simply place ramp against the edge of the door runner and attach.
- Attach your ramp to the threshold or landing using the holes in either the foot section or the flap. Wood screws, sheet metal screws, or concrete bolts may be necessary for this method. It is also possible to use double face tape or caulking to attach the ramp. Foam rubber or other protective material may be put on the bottom of the legs to protect certain floor surfaces.

### MULTI-RAMP APPLICATIONS UNEVEN THRESHOLDS OR SLIDING DOORS

There are situations where a second Threshold Ramp may be needed on the inside of the threshold. If so, then the purchase of a second Threshold Ramp of equal or different height will be necessary. Follow instructions for single ramp application when installing second ramp.

For sliding doors, the flap will not be used. Simply place leg of ramp against edge of door runner and attach per item 'c' in SINGLE RAMP APPLICATION section.

## FIGURE 4

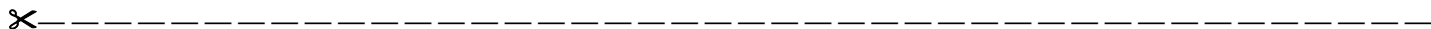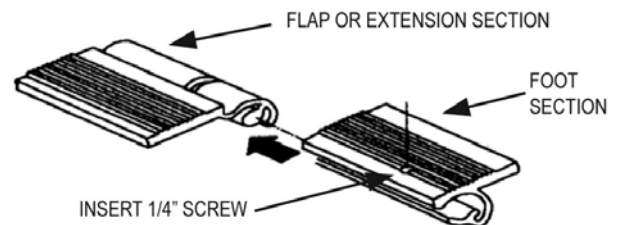

## WARRANTY REGISTRATION

EZ-ACCESS®, A DIVISION OF HOMECARE PRODUCTS, INC. • 1704 B ST NW STE 110 • AUBURN WA 98001-1650

PRODUCT	STOCK NUMBER
NAME	PHONE
ADDRESS	CITY, ST, ZIP
NAME OF DEALER	DEALER'S CITY, ST, ZIP
PRICE	DATE PURCHASED
COMMENTS	
WHAT INFLUENCED YOUR PURCHASE OF THIS PRODUCT?	
<input type="checkbox"/> SAW IN STORE <input type="checkbox"/> RECOMMENDATION OF FRIEND <input type="checkbox"/> RECOMMENDATION OF DEALER <input type="checkbox"/> MAGAZINE AD	
<input type="checkbox"/> OTHER (PLEASE EXPLAIN):	