


## SIDE-KICK

Reorder #	Description	Unit
□ 703455	16"-18" Side-Kick/Footrest Extender Add-On	Each

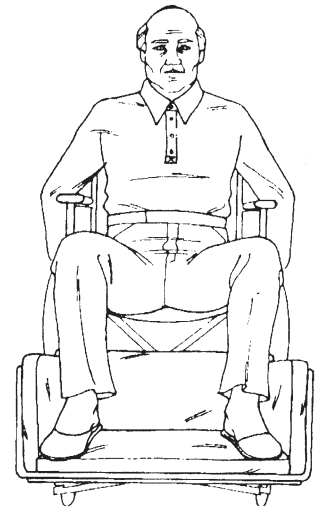

### PURPOSE


The Side-Kick prevents the wheelchair user's feet from slipping off the sides of the footrest extender pad. The vinyl-covered side panels are foam padded for comfort, and they protect feet, ankles, and lower calves against contact with metal wheelchair parts. The Side-Kick is designed to fit Skil-Care's Drop-Stop Footrest Extender, Bi-Level Footrest Extender, and most footrest extenders made by other manufacturers.

### INSTALLATION


1. Remove footrest pad from footrest extender. (Illustration A)
2. Place Side-Kick on base of footrest extender frame. (Illustration B)
3. Place footrest extender pad on Side-Kick. (Illustrations C & D)

### EASY CARE


The Side-Kick frame and vinyl side pads may be cleaned with a spray-and-wipe cleanser. Accumulated lint and other particles may be removed from the Velcro® strips by means of a stiff brush.

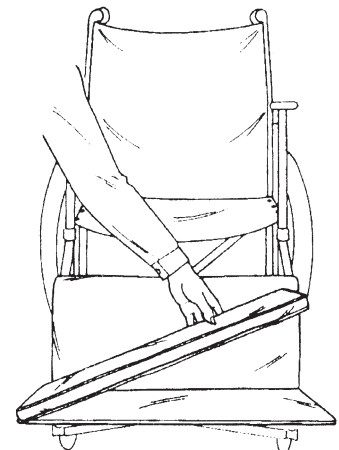


**A**

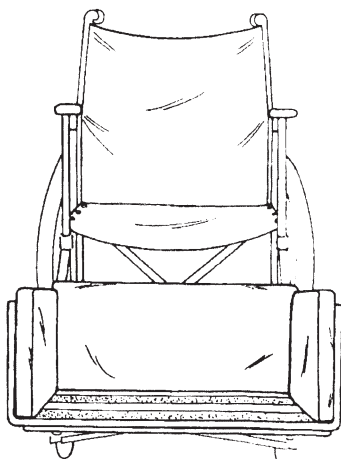

**B**

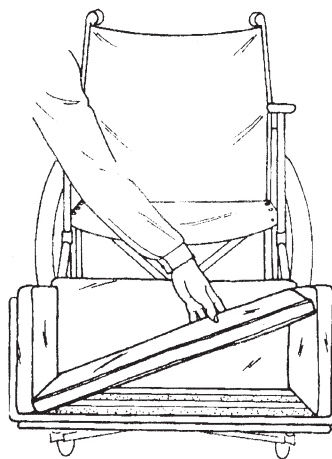

**C**

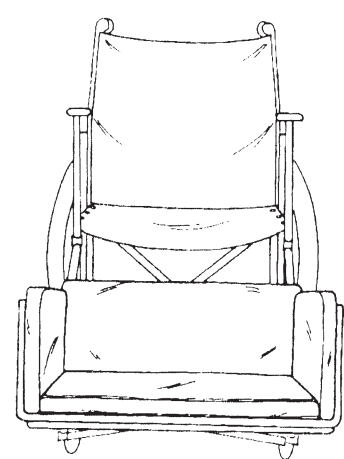

**D**