



Carex Health Brands



**Mobility & Comfort for
your active lifestyle**

Carex Health Brands • Sioux Falls, SD • www.carex.com
42-A22300-02 ©2012



Step N' Rest®

Rolling Walker



Carex Health Brands

1 Check Parts

Rolling walker frame




Backrest



Rear wheels



Handles with brake assembly

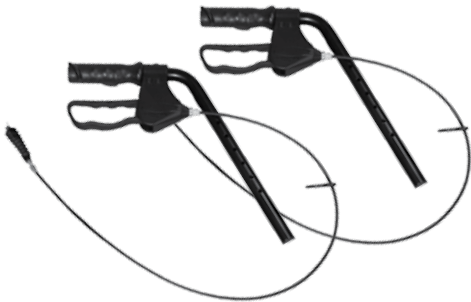

C-clips & hand screws with washers





Front wheels




Note: actual items may vary slightly from what is shown.

2 Brake Assembly

BRAKE may come pre-assembled, if so skip to step 3

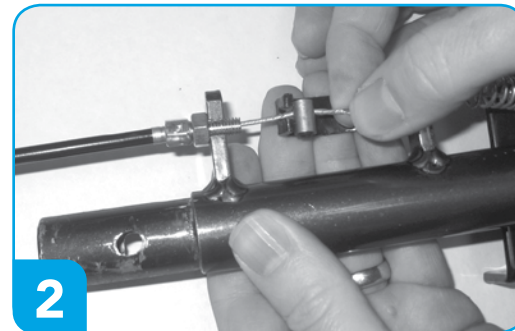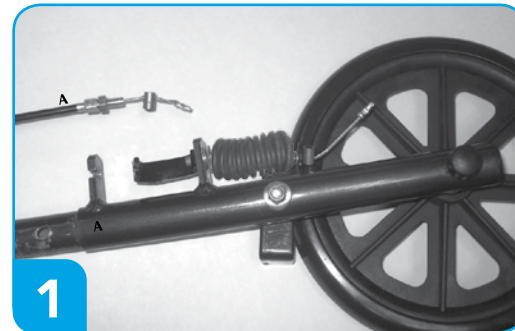

3 Handle Assembly

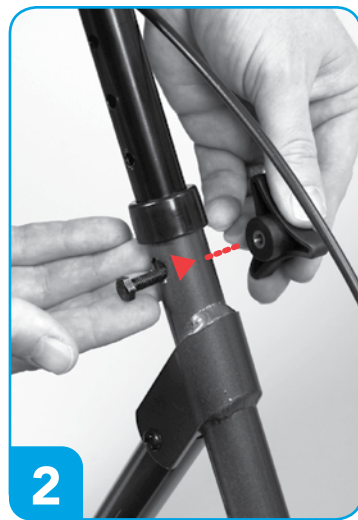

 **NOTE:** Hex shaped bolt should fit onto hex shaped hole in tubing. Hand screw faces out.


Cable should be outside frame.



4 Front & Rear Wheel Assembly



Line up holes.
C-clip in small hole.

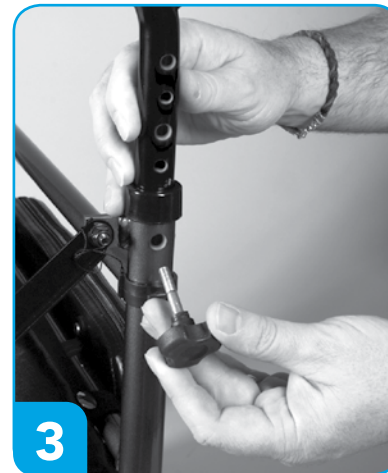

Hand screw in large hole




5 Backrest Assembly & Braking System




Push in side buttons and insert backrest. Flip up backrest for compact storage.




Using the brakes

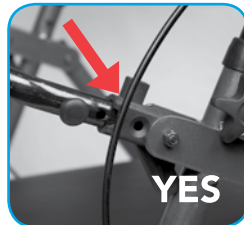

Note: Cable needs to be placed outside seating area and frames.

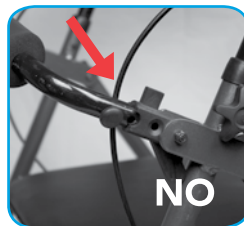

6 Adjusting the Handles & Seat Height

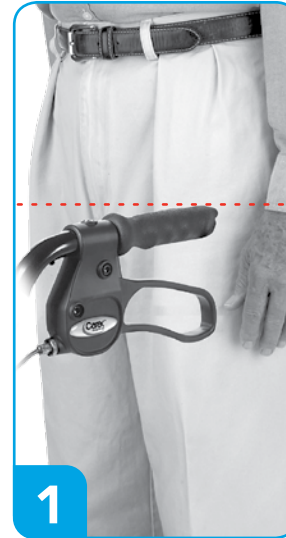

high —
med —
low —