

HandyBar Instructions



The Handybar® inserts into the U-shaped striker plate on your vehicle door frame. The U-shaped striker plate is one of the strongest parts of the vehicle and is designed to keep your vehicle door closed in the event of a side-impact collision.

There is a striker on each of the vehicle's door pillars allowing the Handybar® to be used on both driver and passenger side-doors.

The Handybar® is suitable for individuals weighing up to 350 lbs.



Insert the Handybar® into the U-shaped striker plate located on the driver and passenger-side door frames of the vehicle.



Once inserted, use the Handybar® to support your weight as you move yourself in and out of the vehicle.



Remove the Handybar® before you close the vehicle door.

Ensure your vehicle has two bolts or screws attaching the striker to the door frame. If your vehicle has a light switch in the striker, be sure the Handybar does not interfere with its operation.