



MJM INTERNATIONAL CORPORATION

E118-3TW-FF-SQ-PAIL

ASSEMBLY INSTRUCTIONS


NOTE: A PHILLIPS SCREWDRIVER OR A CORDLESS HAND DRILL IS REQUIRED TO TIGHTEN / SECURE ALL SCREWS.

IDENTIFY COMPONENTS TO START ASSEMBLY




- ☐ SEAT SECTION WITH ARMS, COMMODE, FOLDING FOOTREST
- ☐ PVC BACKREST WITH MESH SLING.
- ☐ BOX WITH CASTERS AND SCREWS
 - 4 CASTERS WITH BRAKES.

STEP 1 – CASTERS

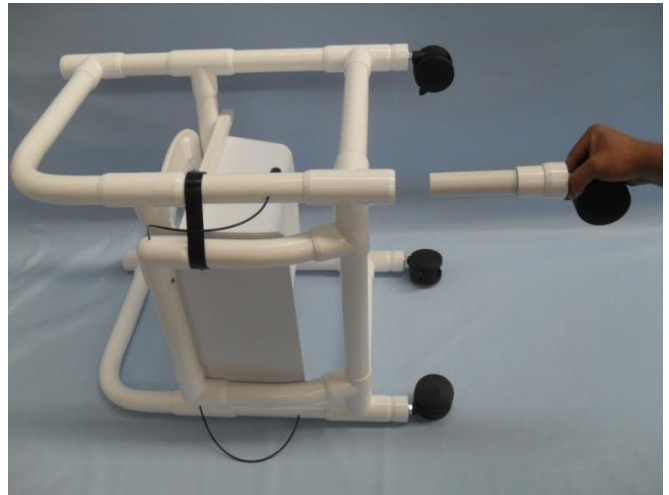

- ☐ INSERT CASTER LEG STEMS INTO BASE SECTION

STEP 2 CASTERS.

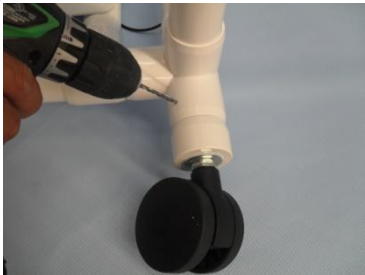

- ☐ DRILL THROUGH LEG PIPES USING 1/8 DRILL BIT AND PILOT HOLES AS REFERENCE.



- ☐ SECURE USING 1/2" RETAINING SCREWS AND PILOT HOLES AS REFERENCE.

STEP 3 BACKREST & MESH SLING



- ☐ INSERT PVC BACKREST INTO T-FITTING LOCATED ON ARM SECTIONS.



MJM INTERNATIONAL CORPORATION

STEP 4 BACKREST & MESH SLING



- SECURE USING 1/2" RETAINING SCREWS AND PILOT HOLES AS REFERENCE.

STEP 5 FOLDING FOOTREST



- SECURE WITH VINYL STRAP.

STEP 6 PRODUCTS COMPLETELY ASSEMBLED



- E118-3-FF-SQ-PAIL



- E118-3-FF-BB-18-SQ-PAIL



- E118-3-FF-LSB-18-SQ-PAIL

NOTE: CASTERS, PVC BACKREST MUST BE CHECKED AND SECURED PRIOR TO USING SHOWER CHAIR.
WARNING NOT SECURING ALL THE ABOVE CAN CREATE A SERIOUS SAFETY HAZARD.