



# Days™ Standard Adjustable Cane with Round Handle


0815-61-760

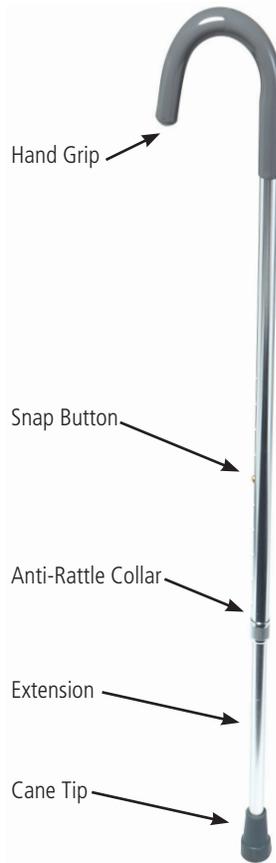

## Proper Cane Height

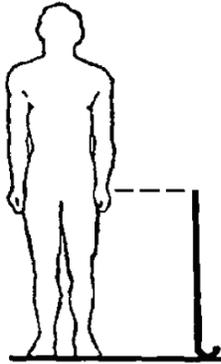

## Set Height Adjustment

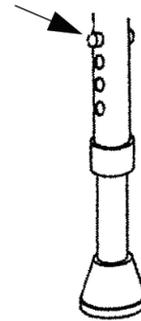

Inspect the product for damage that may have occurred during shipping. If a defect is found, call us immediately.

**Read the following instructions. If they are not clear, consult a professional caregiver or technician.**

Ask a therapist or physician for directions on proper adjustments and specific usage for you.

### Proper Cane Height

Wear the type of shoe you will be wearing most of the time to get the most comfortable fit when using the cane. On a firm, flat surface stand as straight as possible and let your arm hang naturally by your side. The proper height will be adjusted on cane so the top of handle is positioned it touches the palm where it meets the inside of wrist.

This measurement will determine which adjustment hole to use to achieve the desired cane height between 30" to 39".

### Set Height Adjustment

Loosen the anti-rattle collar by turning it counter-clockwise. Press the snap button and slide extension to hole that will achieve the desired cane height. Release the button when it is all the way through the hole on the handle end of cane. You will hear the button "click" once it's secured in the hole. Repeat this process until the proper cane height is reached. Turn the lock knob clockwise to tighten.

### Precautions

**Do not exceed** user weight capacity of 250 lbs. The cane cannot support the full body weight; use it only to increase stability and assist the user when walking.

**Before using, double check that the snap button protrudes all the way through the adjustment hole, so that you know the height of cane is locked.**

Periodically inspect rubber tips for wear and tear. If tips are damaged, replace them immediately.

### Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



PATTERSON  
MEDICAL

www.pattersonmedical.com

© August 2015 Patterson Medical Products, Inc.  
081561760LIT

Patterson Medical  
28100 Torch Parkway, Suite 700  
Warrenville, IL 60555-3938  
Phone: 800-323-5547  
Direct Phone: 630-393-6000  
Fax: 800-547-4333