09 116 6883

Instruction Sheet 

 **orbi-turn**

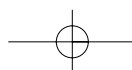
Please retain these instructions for future reference.

**Manufactured for: Homecraft Rolyan**, A Patterson Medical Company,  
Nunn Brook Road, Huthwaite, Sutton-in-Ashfield, Nottinghamshire, NG17 2HU, UK.  
**UK:** Tel: 01623 448 702 Fax: 01623 557 769 [www.homecraft-rolyan.com](http://www.homecraft-rolyan.com)  
**International:** Tel: +44 1623 448 706 Fax: +44 1623 448 784

**Sammons Preston Rolyan**, 4 Sammons Court, Bolingbrook, IL 60440-5071, USA.  
Tel: 1-630-226-0000 Telefax: 1-630-226-1390

**homecraft**  

# Orbi-Turn


Before using any manual handling equipment it is essential that you read and fully familiarise yourself with the operating instructions. Equipment should only be used by appropriately trained persons who have received regular updated moving and handling training.


The Orbi-Turn has been designed for carers to facilitate standing and seat-to-seat transfers, whilst minimising the risk of injury to themselves and the patient.

## ASSEMBLY AND FOLDING INSTRUCTIONS

The Orbi-Turn is supplied flat packed (**see figure 1**).

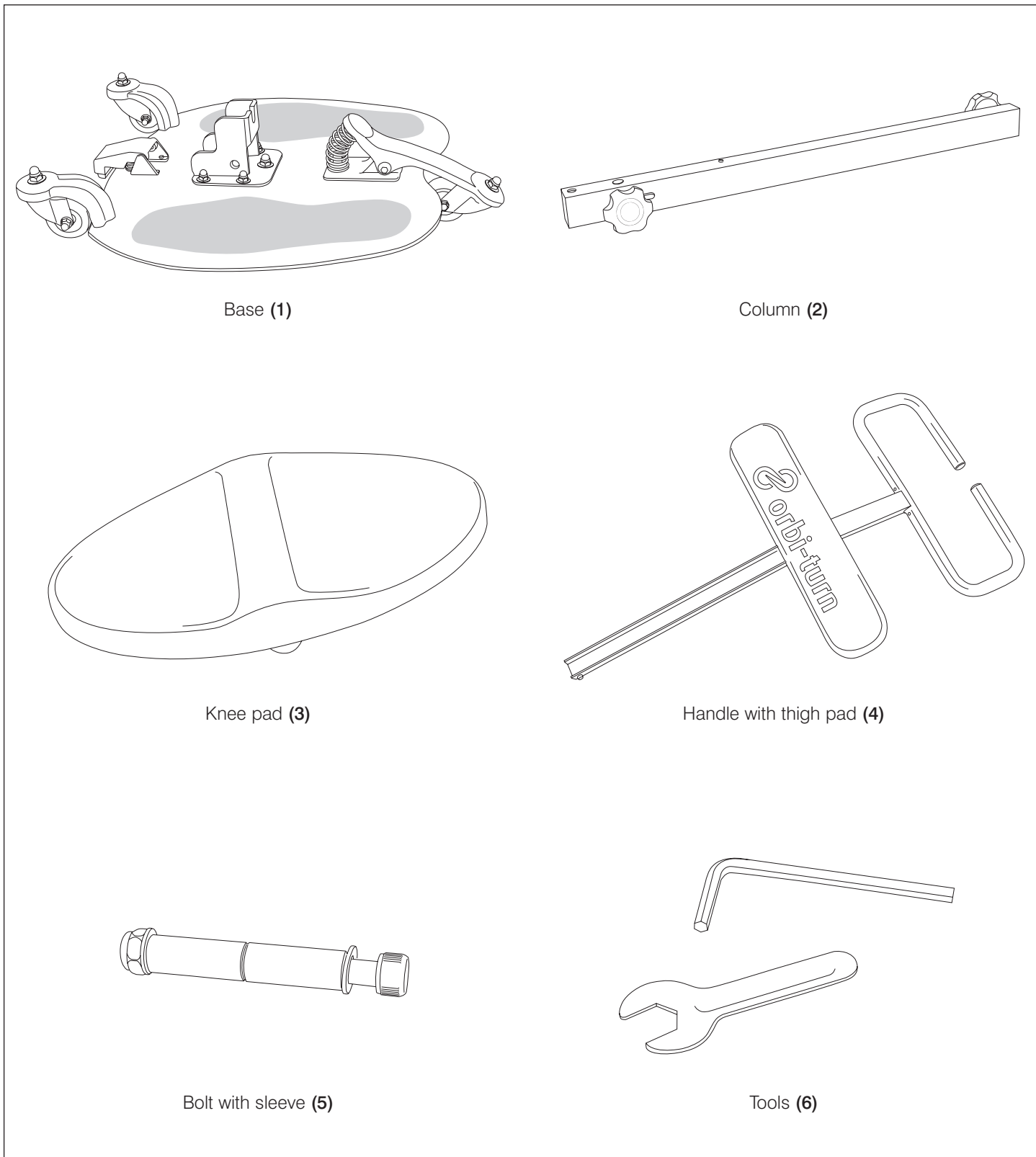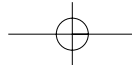**Figure 1** - Line drawings of assemblies:

### To assemble the column to the base:

1. Identify the parts and lay the base (1) on the floor so that it rests on the three castors.
2. Unscrew and remove the nut and washer from the bolt with sleeve (5).
3. Put the column (2) into the base bracket so the slot and folding handwheel is in the lower front position.
4. Line up the cross hole in the column tube with the hole in the angle bracket and push the bolt and sleeve through both parts. With the bolt with sleeve in place, replace the washer and nut and tighten them fully with the tools (6) provided.
5. Loosen the folding mechanism handwheel and raise the column into the vertical position. The folding handwheel will drop into the front slot of the base bracket. Tighten the folding handwheel to secure the column in place.

### To assemble the knee pad:

1. Remove the handwheel at the top of the column.
2. Slide the U shaped bracket over the column ensuring the rubber ferrule is at the bottom and the knee pad is facing forwards.
3. Once the knee pad is at the desired height, tighten the knee pad handwheel to secure it in place. Replace the handwheel at the top of the column.

### To assemble the handle bar:

1. Press in the bullet catch that protrudes from the bottom of the handle (4) tube.
2. Slide the handle tube into the main column tube. The bullet catch will engage in the hole in the side of the main column tube.  
**Please Note:** the handle should not be extended beyond this point during use.
3. Press the bullet catch in and lower the handle to the desired height. Holes in the handle tube provide a positive location for the handwheel. Ensure the handwheel is in a hole and tighten to secure the handle.

### To fold the Orbi-Turn:

1. Loosen the folding mechanism handwheel.
2. Raise the handwheel to allow it to clear the base bracket.
3. Lower the column and handle to the floor.
4. To return the handle and column to the upright position, raise the column into the vertical position. The folding handwheel will drop into the front slot of the base bracket. Tighten the folding handwheel to secure the column in place.

## CLEANING AND MAINTENANCE INSTRUCTIONS

Inspect the turner before each use to ensure that it is free from damage and excessive wear. Look particularly for dents, cracks, loose parts or other damage that may affect the integrity of the product.

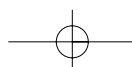
It is recommended that surfaces are wiped clean using a wet cloth with a mild detergent or disinfectant. Ensure that the unit is dry before use.

## TECHNICAL SPECIFICATIONS

Constructed from steel for durability and strength.

<b>Unit weight:</b>	15kg
<b>Base diameter:</b>	400mm
<b>Bottom handle:</b>	
Lowest position	815mm
Highest position	1175mm
<b>Top handle:</b>	
Lowest position	920mm
Highest position	1315mm
<b>Kneepad (Mid point):</b>	
Lowest position	235mm
Highest position	510mm
Maximum user weight	

**Please Note:** as the weight of the patient increases the apparent assistance given by the weight and low centre of gravity of the product will reduce. Consequently the force that the carer must exert will increase.

## INSTRUCTIONS FOR USE

Ensure that the folding mechanism and handle bar are secure and locked in place before using the Orbi-Turn.

A full risk and environmental assessment must be carried out by an appropriately qualified person prior to using this piece of equipment.

1. Ensure that the patient is positioned ready for transferring. Manoeuvre the Orbi-Turn in front of the seated patient. Place the patient's feet on the textured turn plate (**see figure 2**).
2. Adjust the knee-pad just below the tibial tuberosity to facilitate normal biomechanical hip and knee movement.
3. Ask the patient to grip the lower handle bar while you place your foot on the brake for security and to counter balance the force (**see figure 3**).
4. Place your hands on the top handle and ask the patient to stand up. When standing, the patient can safely lean against the thigh pad for increased comfort and security during the transfer (**see figure 4**).
5. Once the patient is standing safely, remove your foot from the brake and gently rotate the patient on the Orbi-Turn (**see figure 5**). Ensure that you step round with the Orbi-Turn to avoid spinal rotation.
6. Once in the desired position, operate the foot brake and ask the patient to slowly sit down while counter balancing their weight (**see figure 6**).
7. With the transfer complete remove the Orbi-Turn. Tilting back onto the rear castors will enable the Orbi-Turn to be easily transported away and stored (**see figure 7**). When the handle bar has been raised to the maximum height, a bullet catch will prevent further extension. The handle bar should only be pulled out further for cleaning or transportation.

Figure 2

Figure 3

Figure 4

Figure 5

Figure 6

Figure 7

## DO'S AND DONT'S

### Do.....

- Do** read the user instructions thoroughly before using the Orbi-Turn.
- Do** ensure that appropriate and regular moving and handling training is received by those using the equipment.
- Do** ensure that a full risk assessment is carried out before any transfer is attempted.
- Do** wear appropriate footwear and clothing.

### Do not...

- Do not** exceed the weight limit of the Orbi-Turn - 190kg/30 stone
- Do not** use the Orbi-Turn for transfers if the folding mechanism is not locked into place.
- Do not** use the Orbi-Turn on someone who is unable to weight bear or co-operate with the planned transfer.
- Do not** transport the patient while they are stood on the Orbi-Turn.

The Orbi-Turn has been designed as a standing and turning aid only.

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