Rolyan® Graded Pinch Exerciser

A196-6

Pinchpins

A196-100 through 600

INDICATIONS

- To increase strength and coordination in an upper extremity

CONTRAINDICATIONS

Not in the presence of:

- fixed deformities of the upper extremity
- acute carpal tunnel syndrome
- Guyan tunnel syndrome
- unhealed fractures of the upper extremity
- unhealed flexor tendon repairs

PRECAUTIONS

- Open and close the pinchpins carefully to avoid overstressing or pinching the fingers.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

CARE

Clean the pinchpins and base with a hard-surface disinfectant.

INSTRUCTIONS FOR USE

The resistance of each color of pin varies depending on how far it is opened. The following is the approximate resistance when the pin is open just far enough to put it onto the largest bar. The tip of the pin will be open $\frac{3}{8}$ " (9.5 mm).

Yellow	1 lb. (0.45 kg)
Red	2 lb. (0.9 kg)
Green	4 lb. (1.8 kg)
Blue	6 lb. (2.7 kg)
Black	8 lb. (3.6 kg)

The user pinches open a pin and puts it onto one of the three horizontal rods. The exercise can be progressed by increasing the rod size as well as the resistance of the pins.

To increase shoulder range of motion, the vertical rod can be inserted into a hole in the highest side of the base. It comes in three pieces, which twist together. The section with the orange protective tip should be at the top.

The pinchpins can be used independently of the Rolyan Graded Pinch Exerciser to continue the treatment program at home.

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Patterson Medical Products, Inc.
Patterson Medical, Ltd.