Rev. 0 - 02/11_108741

Bauerfeind AG
Weissendorfer Strasse 5
07937 Zeulenroda-Triebes
Germany
Phone +49 (0) 36628-66-40 00
Fax +49 (0) 36628-66-44 99
E-mail info@bauerfeind.com

GenuTrain®

Aktivbandage
Active support
Orthèse active
Vendaje activo

Motion is Life: www.bauerfeind.com

Motion is Life: www.bauerfeind.com

Material
Polyamid (PA)
Cawiton*
verzinkter Federstahldraht
Elastodien (ED)
Polyurethan (PUR)
Elastan (EL)
Viskose (CV)
Baumwolle (CO)
Polyethylen (PE)
Polyester (PES)

*Cawiton PR6553 A

Material
Polyamid (PA)
Cawiton*
Galvanized spring steel wire
Elastodiene (ED)
Polyurethane (PUR)
Spandex (EL)
Rayon (CV)
Cotton (CO)
Polyethylene (PE)
Polyester (PES)

*Cawiton PR6553 A

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EN Dear customer,

thank you for choosing a product from Bauerfeind. GenuTrain® is a high-quality medical product.

Please read through these instructions for use carefully and if you have any questions contact your doctor, medical retailer or our Technical Service department.

GenuTrain®, the active knee support, produces its effect during movement. During movement, the shaped viscoelastic ring compresses and massages the soft tissues. The resulting enhanced circulation in the joint capsule and promotion of lymph drainage lead to reduced swelling, pain relief and improved function while, at the same time, taking the strain off the kneecap.

The three-dimensional knit ensures an anatomically perfect fit for the GenuTrain. The compression-reduced support edges stop the support cutting into the leg.

 If the product is not used properly any claim for warranty will expire.

Indications

- Irritation after an injury, operation or overexertion of tendons, muscles or ligaments
- Degenerative changes in the knee (osteoarthritis)
- Joint inflammation (arthritis)
- Feeling of instability

Side effects

Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses¹ applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical advisor:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Varicose veins.
3. Impaired sensation and circulatory disorders of the legs (e.g. diabetes mellitus).
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Fitting GenuTrain® (Illustrated)

- Firmly grip the top of the support with both hands (one either side of the lateral spines) at the level of the ring-shaped donning aid and pull over the knee joint until the knee cap is free from pressure and is surrounded by the silicone insert. **1 2 3**
- Adjust as required until a snug fit is achieved. **4**
- In contact sports a protective covering should be applied.

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

Removing GenuTrain®

To remove the support, pull down at the bottom of the support near the stays. **5**

Important Information

- GenuTrain® is a prescribed product that should be applied under a physician's guidance. GenuTrain® should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product is intended to be fitted only once and to one patient.
- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical advisor without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical advisor.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.

Product care

 All Train products² can be machine washed separately at 30 °C with liquid detergent. Please note the following when washing your Train products:

1. For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.
2. If possible, also remove pads/inserts and their terrycloth covers before washing. We recommend using a mesh bag (spin cycle: maximum of 500 revolutions per minute). Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric.

Please wash your Train product regularly so that the compression of the knitted fabric can be retained.

This product has been tested by our in-house quality management system. However, should you have any complaint, please contact our local expert adviser or our customer service department.

Version: September 2010

² Standardized washing tests show that after 50 washing machine cycles full functionality of the Train active support is retained. (Liquid detergent was used.)

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