


Days™ Two-Button Folding Walker


Adult 0815-61-687
Youth 0815-61-711

Adult with 3" Wheels 0815-61-695
Youth with 3" Wheels 0815-61-729

Adult with 5" Wheels 0815-61-703
Youth with 5" Wheels 0815-61-737

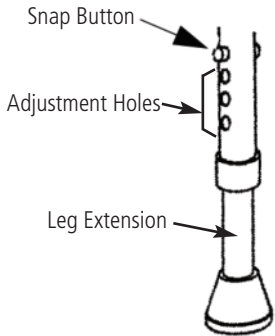

Set Height Adjustment

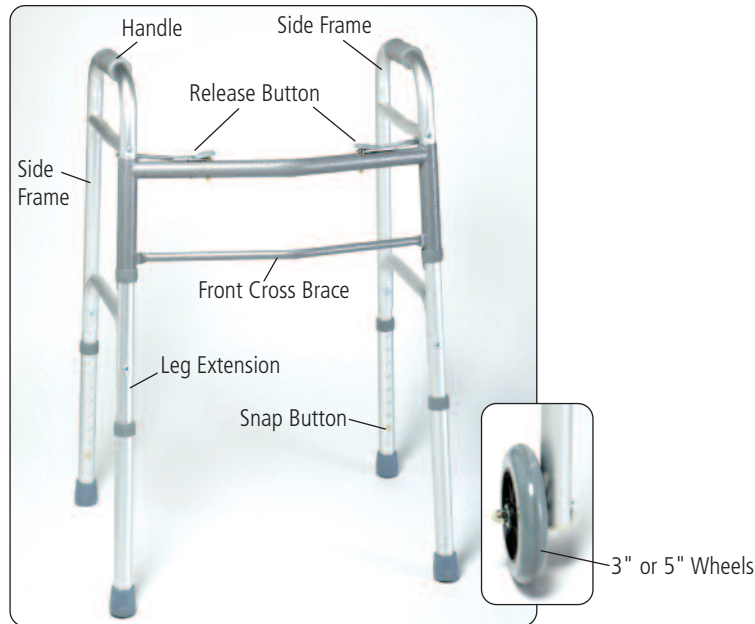

Pre-Assembly

Inspect the product for damage that may have occurred during shipping. In the case of defects, contact us immediately.

Read the following instructions before attempting usage. If the directions are not clear, contact a healthcare professional or technician.

To achieve the proper adjustments and guidelines for use, consult with a therapist or other medical professional.

Opening/Folding the Walker

To open, push down the release buttons and swing the side frames open until the buttons pop back up. At the same time, you will hear a "click," indicating that the walker is open all the way. The open position will be securely locked until the release button is used again for folding. Press the buttons and fold in one side of frame and then the other.

Adjust Leg Extensions

Determine the desired overall height of the walker:
From 32"–39" for Adult
From 25"–32" for Youth

The height of walker should be at the wrist when the user's arms are down.

Select the adjustment hole on the leg of the frame that will achieve the desired height. Push the snap button on the leg of the frame, slide the leg extension to the chosen hole and release the button to lock in place. Repeat steps for each leg extension.

Confirm that the chosen adjustment holes are at the same height and that snap buttons are fully positioned through the holes.

Precautions

Do not exceed the maximum user weight of 300 lbs. The cane aids stability and walking; do not use it to support full body weight.

Before each use: ensure snap buttons protrude all the way through the holes. All leg extensions MUST be set at the same height for safe operation.

Before using, double check that walker has been fully opened and is level to the floor or ground.

The walker is not designed to hold items. Hanging objects on the walker could tip it over and potentially causing personal injury or damage.

Periodically inspect rubber tips for wear and tear. If tips are damaged, replace them immediately.

Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



Patterson Medical

1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310



Authorized European Representative Patterson Medical, Ltd.

Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom
Phone: 08444 124 330
Fax: 08448 730 100
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784



© May 2012
Patterson Medical Products, Inc.
081561687LIT