

Patterson Medical Positioning Wedge

0815-70-308 Small, 7" x 12" x 24" Sold in Pairs (2)

0815-70-316 Large, 7" x 12" x 34" One (1)

Indications

- May be used to provide trunk stabilization.
- Use to help maintain side-lying bed position for patients who need to off-load coccyx/sacrum area. Angled design provides better positioning than pillows.
- Use under knees when patient is lying on back to provide slight knee flexion, helping to off load lower back pressure.

Contraindications

- Single patient use

Application

- The wedge is placed under patient's back, just below the shoulder to maintain side lying. Wedge can also be placed under patient's knees while patient is lying on back to maintain knees in flexion. Nonskid bottom has to be against bed surface, never against patient.

Washing Instructions

- Surface can be cleaned using a damp cloth and cleanser. Follow facility protocol for disinfecting wedge. Use a non-chlorine disinfectant only. Do not immerse in water.

Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



Patterson Medical

1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310



Authorized European Representative Patterson Medical, Ltd.

Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK



© January 2012
Patterson Medical Products, Inc.
081570308LIT

United Kingdom
Phone: 08444 124 330
Fax: 08448 730 100
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784