

Carpal Gel Wrist Support

APPLICATION INSTRUCTIONS

Warning: Do not place gel pad on an open wound. If you experience skin irritation, pain or discomfort while using this product, remove immediately and consult your healthcare provider.

The use of this product will not prevent injury.

Note: Plastic stay may be removed if advised by health-care provider.



Release all hook and loop closures. Remove the protective plastic sheet from the gel pad. Position gel pad so it aligns with curvature of the palm/wrist area.



Wrap wrist strap snugly around wrist and fasten hook to wrist support.



Pull thumb strap between thumb and forefinger. Fasten hook to wrist support.

 LOHMANN & RAUSCHER

6001 SW Sixth Avenue
Suite 101
Topeka, KS 66615-1004
Phone: 800-279-3863, 785-862-1100
e-mail: office@us.LRMed.com
Website: www.lohmann-rauscher.us

700160/R0912

WASHING INSTRUCTIONS



- Do not remove gel pad.
- Hand wash in warm water. Do not use soap.
- Air dry. Do not tumble dry.
- Do not bleach, do not iron, do not dry clean.

Sprinkle pad with talcum powder if gel becomes tacky.