

What Should You Have?

When you open your package of Lock Laces®, you should find the following:

- 2 48" Elastic Shoelaces
- 2 lock devices
- 2 cord clips



You will also need:

- 1 pair of shoes
- 1 pair of scissors

Alright, now that we have everything we need, let's get started!

Step 1: Replace Your Laces

Put your feet in the shoes. We always recommend keeping your feet in the shoes throughout the installation process to ensure a proper fit.



Now remove your old shoelaces and lace up with your new elastic shoelaces.

Step 2: Lock-In

Grab a lock device (the piece that says “LOCK LACES®” on it). Press down on the top of the lock to open the passages, and thread the ends of the laces through the passages with your other hand.



Once the laces have been threaded through, press down on the top of the lock again and slide the lock to the tongue of the shoe. Release to secure the lock device into place.



Step 3: Trim to Fit

Before trimming the excess lace, make sure your laces fit snugly but are not too tight. We recommend wiggling your foot in the shoe or, better yet, standing up and walking around the room to make sure they are at a comfortable length. If your laces are too tight, slide the lock toward the toe of the shoe slightly, readjust the laces and repeat.



Once your laces fit comfortably, use a pair of scissors to trim off some of the excess lace. We recommend cutting about 2-3 inches below the lock device to leave room for future adjustments.

Step 4: Secure

Finally, slide the cord clip onto the ends of the laces and align the trimmed ends with the teeth inside the cord clip. It is very important that you align the ends with the cord clip properly to ensure that the cord clip stays securely attached. Then, simply snap the cord clip together to secure.

