

CORFIT® ADVANTAGE AP

LUMBOSACRAL SPINAL SUPPORT

INSTRUCTIONS FOR USE

Prior to putting support on: Attach posterior panel (Core logo facing up) to center back of support. Insert plastic anterior panel into mesh pocket. Unload and loosen side pulls by attaching to sides of the support.



Position support across back with bottom edge at the top of Coccyx.



Insert hands into closure assists. Close right panel over left and press to fasten.



Insert thumbs into pull assists and stretch outward, then forward. Fasten to front of support.

Top pull adjusts compression to lumbar region; bottom pull adjusts compression to sacral region.

For additional compression, readjust support while lying on back, knees bent and feet on floor.

⚠ WASHING INSTRUCTIONS

Hand wash in cold water using mild detergent. Hang dry.



www.CoreProducts.com • 800.365.3047
808 Prospect Ave. • Osceola, WI 54020 USA

To determine size, measure around hips.

Small - 6" 27 - 35" 69 - 97 cm	Med/Lg - 8" 32 - 43" 81 - 109 cm	XL - 9" 40 - 52" 102 - 132 cm	2X - 10" 46 - 58" 117 - 147 cm	3X - 12" 52 - 64" 132 - 163 cm
---	---	--	---	---