AIRCAST® AIRSELECT™ ELITE

ENGLISH2
ESPAÑOL4
DEUTSCH.....6
ITALIANO.....8
FRANÇAIS 10
NEDERLANDS..... 12
SLOVENČINA 14
DANSK 16
SVENSKA..... 18
SUOMI 20
ČEŠTINA 22
PORTUGUÊS..... 24
NORSK 26
简体中文.....28
日本語 30

AIRCAST®

DJO, LLC
A DJO Global Company
1430 Decision Street
Vista, CA 92081-8553 • USA

T 760.727.1280 • F 760.734.5608
DJOglobal.com

©2012 DJO, LLC

01E970_B

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE / INDICATIONS:

The ELITE walking brace provides compressed air support for stable fracture of foot and/or ankle, severe ankle sprains, and post-operative use. Effective in fracture healing and the management of edema (swelling).

WARNINGS AND PRECAUTIONS:

Do not over-inflate aircells. Improper aircell inflation may cause significant skin irritation, especially in people with diabetic neuropathy (a diabetic nerve condition). Check your skin frequently for "hot spots" and skin irritation. If you see any skin changes or feelings of discomfort, reduce the aircell compression. Do not use this device if you cannot feel sensations whether due to post-op anesthesia, nerve, skin or other conditions.

Use caution when walking on slippery or wet surfaces to avoid injury.

APPLICATION INFORMATION:

1) PREPARE BRACE –

- Unfasten straps, remove front panel, and open toe cover and foot flaps.

2) APPLY BRACE –

- First, put on sock included with the brace.
- While seated, place foot in brace positioning the heel against the back of the brace. Wrap foot flaps over the foot and the toe cover around toes (Fig. A).
- *Replace front panel:* Position front panel so the Aircast logo is over the toes and the foot panel is situated inside the brace. The upper part of the panel should be secured outside the brace (Fig. B).
- Secure straps from bottom to top (Fig. C). Tighten until snug and comfortable.







3) ADJUST AIRCELL COMPRESSION –

The ELITE walking brace aircells are initially deflated

- Begin inflation on injured side first. Turn the SELECTOR found on the right-side of the brace to the appropriate number (Fig. D). Number 1 inflates the aircell on the right-side, Number 2 inflates the aircell on the left-side and Number 3 inflates the back aircell.
- To inflate aircell, press and release **large** inflation pump found on the top right-side of brace (Fig. E). Pump until aircell feels snug.
- To inflate other aircells, rotate SELECTOR to the appropriate number and then press and release large inflation pump. Pump until aircell feels snug.
- In the event of over-inflation, deflate aircells.

4) DEFLATE AIRCELLS –

- To deflate aircells, turn the SELECTOR found on the right-side of the brace to the proper aircell number (Fig. D).
- Press and hold finger on the **small** release button found on the top right-side of brace (Fig. F).
- Deflate until pressure is comfortable.

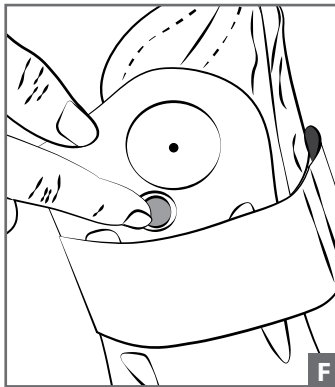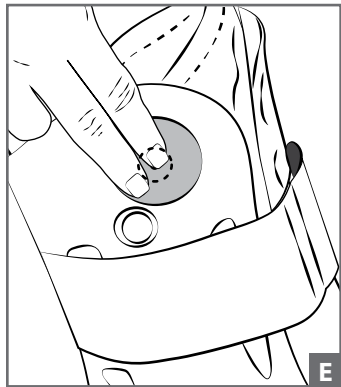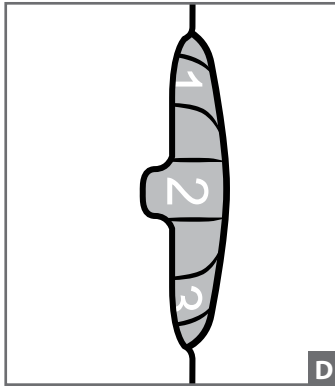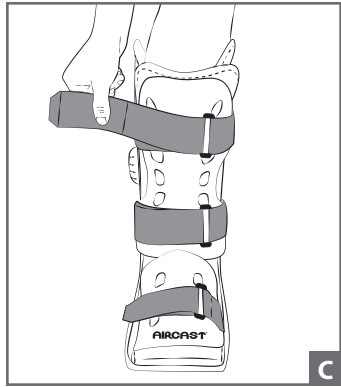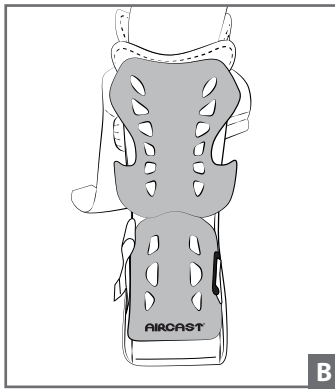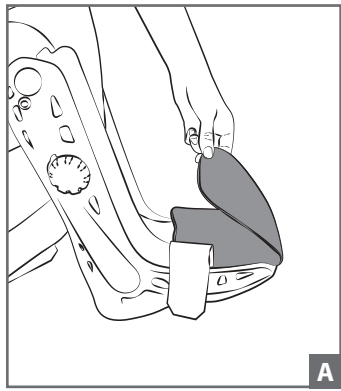NOTE: Deflate aircells **before removing brace** for proper fit when reapplying.

NOTE: Higher pressure provides more support when walking, lower pressure is more comfortable when sitting or reclining. Pressure can also be adjusted by tightening or loosening straps.

NOTE: The shell can be trimmed with a cast saw or cast scissors to accommodate larger calves.

CLEANING INSTRUCTIONS:

THE FOAM LINER MAY BE HAND-WASHED IN 86°F/30°C WATER WITH MILD DETERGENT AND AIR-DRIED. DO NOT PUT IN DRYER OR USE OTHER HEAT SOURCE TO DRY.

HIGH ALTITUDE:

AT HIGH ALTITUDES THE AIRCELLS WILL EXPAND BEYOND THEIR OPTIMAL LEVEL. ADJUST AIRCELL PRESSURE BY TIGHTENING OR LOOSENING THE STRAPS.

WARRANTY:

DJO Global, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

LATEX FREE

INTENDED FOR USE ON A SINGLE PATIENT

MATERIAL COMPONENTS

Shell: POLYPROPYLENE

Front Panel: HIGH DENSITY POLYETHYLENE

Straps: NYLON

Lined Aircells (Anterior and Posterior): POLYURETHANE w/ POLYESTER FABRIC

Inflation Pumps: TPU (THERMOPLASTIC POLYURETHANE)

Liner: NYLON/POLYURETHANE/POLYESTER (OUTER/FOAM/INNER MESH)

Binding: POLYESTER

Outer Sole / Insole: EVA/RUBBER

Selector dial: NYLON

Sock (Elite): ACRYLIC/COTTON

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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