
TheraBite®

JAW MOTION REHABILITATION SYSTEM™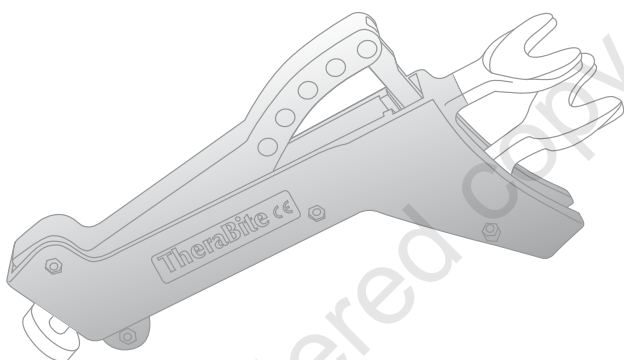

Instructions for use

Region 1

ATOS

MEDICAL

Leading the way in ENT

Read
instructions
before use

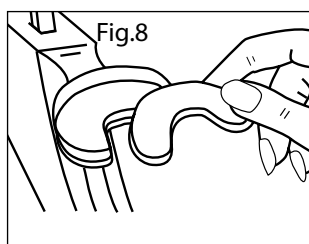
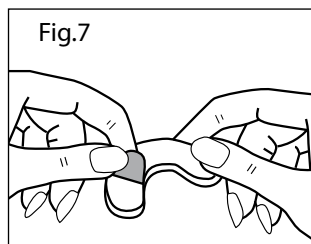
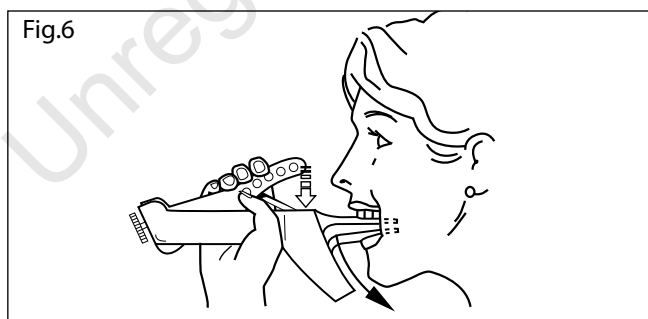
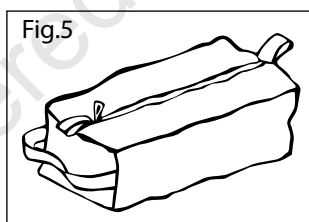
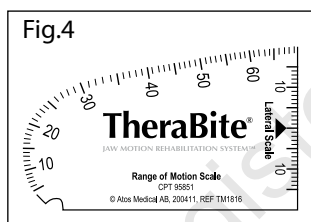
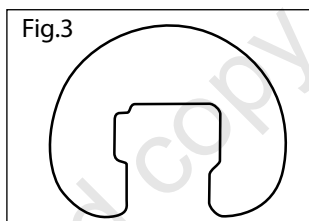
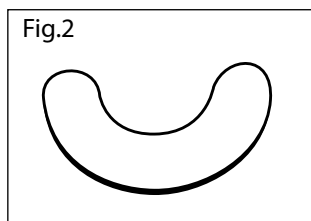
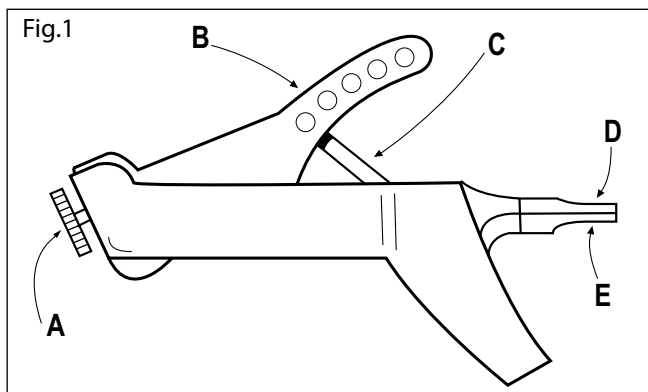
Fig.9a

Fig.9b

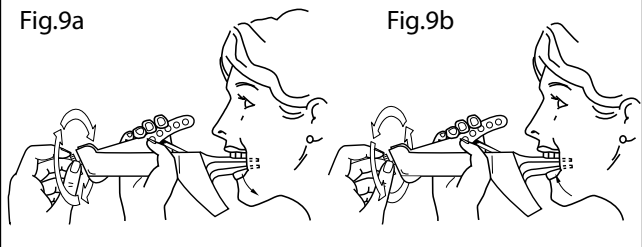
Fig.10a

Fig.10b

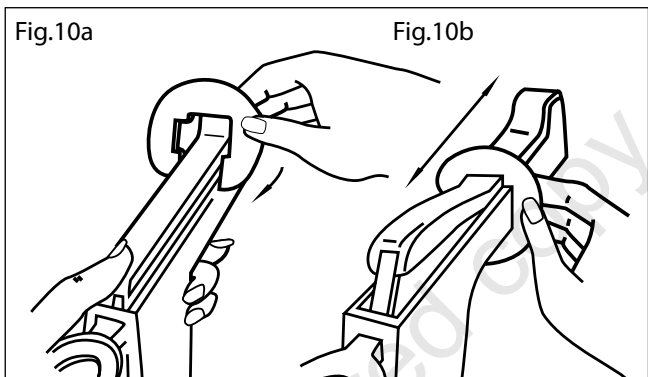
Fig.11

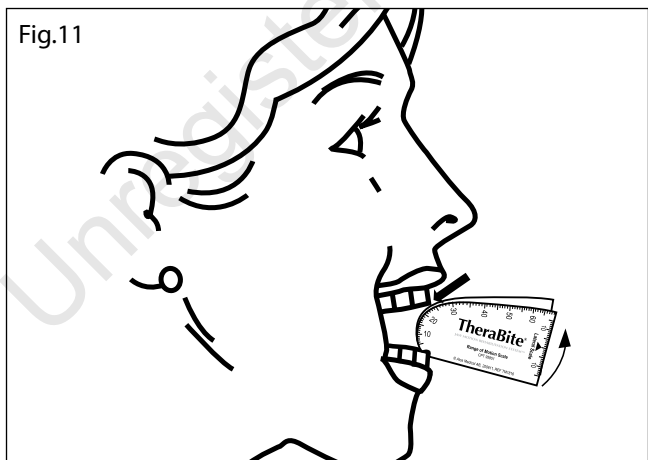
Fig.12a

Fig.12b

Fig.12c

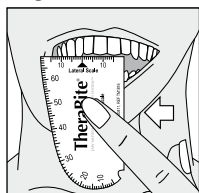
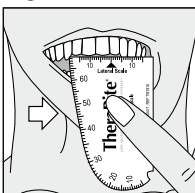
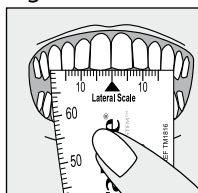
Fig.13a

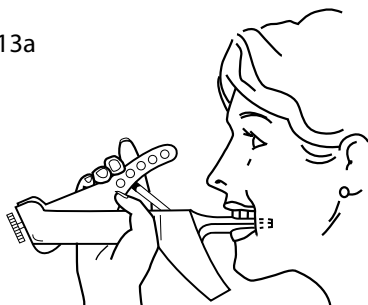
Fig.13b

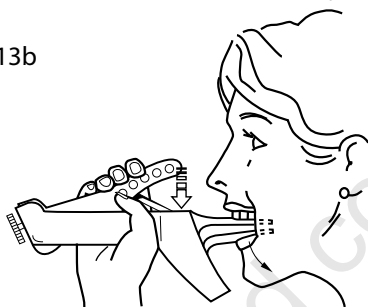
Fig.13c

Fig.14

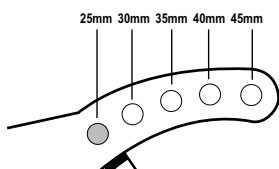
Fig.15a


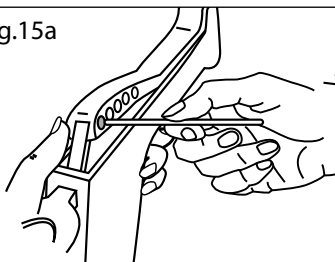
Fig.15b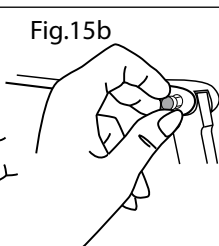

Unregistered copy

LANGUAGES

ENGLISH.....	8-17
DEUTSCH.....	18-28
NEDERLANDS	29-39
FRANÇAIS.....	40-50
ITALIANO.....	51-61
ESPAÑOL.....	62-72
PORTUGUÊS.....	73-83
עברית	84-93
ORDERING INFORMATION.....	94

Content

PART I:	
GENERAL AND USER INFORMATION.....	9
PART II:	
INSTRUCTIONS FOR THE CLINICIAN.....	16
PART III:	
ADDITIONAL INFORMATION.....	17

PART I: GENERAL AND USER INFORMATION

General Instructions

This User Manual contains general safety, operating, maintenance, and care instructions for the TheraBite® Jaw Motion Rehabilitation System™. Both clinicians and users should carefully review this User Manual before prescribing or using the TheraBite system. Users who have any questions about the safety, operation, maintenance, care, or other use of the TheraBite system after reviewing the User Manual should consult with their clinician.

This User Manual, which is provided to all users of the TheraBite system, may be revised periodically and therefore must be reviewed carefully each time the device is prescribed and used.

Before Use

The TheraBite system should not be used in any manner until the patient has consulted with, and received proper instructions from, his or her treating clinician.

Before the TheraBite system may be used for the first time, the maximum opening of the TheraBite device appropriate for each individual user must be determined and set by the user's clinician.

Prior to each use, carefully inspect the product to ensure that it is undamaged and that all components, parts and accessories are complete and in proper condition. Do not use the TheraBite system if the product, or any of its components, parts, or accessories, show any signs of cracking or other structural damage.

Intended use / Indications

The TheraBite system is indicated for use in individuals who are restricted in their ability to open their jaws (trismus or jaw hypomobility).

The TheraBite system is indicated for use in individuals who experience pain in the joints and/or muscles of the jaw.

The TheraBite system may also be used as a rehabilitation tool for postoperative physical therapy of the jaw.

Patients at risk of developing limited jaw motion may consider the TheraBite system for prophylactic use on the advice and under the supervision of a trained clinician.

Prospective users should consult with their clinician to determine whether the TheraBite system is appropriate for their individual conditions.

Contraindications

The TheraBite system should not be used by individuals who have or may have a fracture in the maxilla or mandible (upper or lower jaw) or other weaknesses of the bones of the jaw.

Patients with infections of the jaw, osteomyelitis, or osteoradionecrosis of the jaw should not use the TheraBite system.

WARNINGS

If the user feels unexpected pain at any time while using the TheraBite system, the user should immediately stop using the product and should immediately contact his or her clinician. Injury may occur if excessive force is applied while using the TheraBite system, especially in users with weakened dentition, periodontal disease, weakened bones or joints, or gap-bridging plates. Users should disclose any such condition to their clinician before using the TheraBite system and should exercise extreme caution while using the product.

Before commencing any rehabilitation regimen involving the TheraBite system, all users should receive a thorough, professional evaluation by a clinician who is properly trained in the diagnosis of mandibular dysfunction. All therapeutic programs and rehabilitation regimens involving the TheraBite system should include regular monitoring, review, and evaluation by a clinician.

CAUTIONS

The TheraBite system is intended for single-patient use only. Do not expose the TheraBite system to direct sunlight and excessive moisture. Do not expose the TheraBite system to extreme temperatures.

Do not sterilize the TheraBite system. Sterilization processes may weaken the materials out of which the device is manufactured, as well as the bonding of the mouthpieces, and cause failure under load.

Always handle the TheraBite system with care.

Product description [Fig. 1-5]

The TheraBite Jaw Motion Rehabilitation System consists of:

- The Jaw Mobilizer (1 pc) [Fig. 1] including the following parts:
 - a. Fine Adjustment Knob [Fig. 1a]
 - b. Lever [Fig. 1b]
 - c. Range Setting Arm [Fig. 1c]
 - d. Upper Mouthpiece [Fig. 1d]
 - e. Lower Mouthpiece [Fig. 1e]
- Bite Pad (4 pcs) [Fig. 2]
- Hand-Aid (1 pc) [Fig. 3]
- Range of Motion Scale (30 pcs) [Fig. 4]
- Carrying Bag (1 pc) [Fig. 5]
- User Manual (1 pc)
- Patient Progress Log (1 pc)

CAUTION: The components of the jaw mobilizer are not detachable. Any attempt to detach the parts e.g. the mouthpieces, can weaken the device and cause failure under load.

How does the TheraBite system work?

The TheraBite system is a hand-operated system with adjustable range that is designed to assist users to improve the range of motion and strength of the jaw. The jaw mobilizer is held in the hand during use, with the mouthpieces placed between the upper and lower front teeth. Pressure on the Lever, applied by the user, provides either

opening force or resistance to closing. [Fig. 6]

The TheraBite system utilizes repetitive passive motion and stretching. The jaw mobilizer guides the mandible (lower jaw) along an anatomically correct, curved pathway during opening and closing movements. [Fig. 6]

The TheraBite system can help achieve:

1. Proper jaw opening:
 - by stretching connective tissues
 - by mobilizing joints
 - by strengthening muscles across their full range of motion
2. Reduction in pain and inflammation
 - Passive mobilization of the joint

Research has indicated that mobilization is beneficial in the management of joint disorders. Motion is necessary to prevent the degeneration of cartilage, the layer of tissue that lines most of the joint surfaces. In many cases, movement may stimulate joint healing and reduce pain and swelling.

By stretching and relaxing muscles, gradual mobilization exercises can help individuals with muscle pain to improve their range of motion. Passive motion can also provide therapeutic advantages, especially following an operation.

How to use the TheraBite system

How to put on the Bite Pads [Fig. 7-8]

The TheraBite system comes with self-adhesive bite pads to protect the user's teeth. The clinician may, in the exercise of his or her independent clinical judgment, stack or trim the bite pads for extra protection.

- Peel off the backing to expose the sticky surface. [Fig. 7]
- Attach one bite pad to the Upper Mouthpiece and one to the Lower Mouthpiece. [Fig. 8]
- Apply pressure with your thumb or similar means to ensure sufficient initial adhesion to the mouthpieces. Pressure and warmth under a prolonged period will make the bite pads tackier.
- To enhance adhesion of the bite pads to the mouthpieces, apply the bite pads to the mouthpieces before going to bed and leave them on overnight.

The adhesive bond created by the pressure-sensitive adhesive on the bite pads increases with time, such that the longer the bite pads sit on the mouthpieces prior to using the device, the more strongly they will tend to adhere.

If the bite pads become worn, replace them with the two extra bite pads that are included with the TheraBite system.

For individuals with no teeth (edentulous) or a limited number of teeth, bite pads with thick cushioning for customized trimming can be ordered separately. Bite pads for pediatric use are provided with the TheraBite system for children.

Additional bite pads can also be ordered separately.

CAUTION: bite pads or parts of bite pads that become loose during use might obstruct breathing. Therefore make sure that the bite pads are securely attached to the mouthpieces after trimming and bonding.

How to use the Jaw Mobilizer [Fig. 6]

Hold the jaw mobilizer with the Lever up and with the mouthpieces inside your mouth. Squeeze the Lever gently to move the Lower Mouthpiece downward, following the natural motion of the jaw.

How to use the Fine Adjustment Knob [Fig. 9]

The Fine Adjustment Knob can be used to slow or stop the motion created by the jaw mobilizer.

- Turn the knob all the way in (counter/anti-clockwise) to stop the Lever in the highest position. [Fig. 9a]
- While squeezing gently, turn the knob out (clockwise) to allow the lower mouthpieces to move down slowly. [Fig. 9b]
- The Lever may be released at any time without turning the Fine Adjustment Knob.

How to use the Hand-Aid [Fig. 10]

The hand-aid is a tool designed to hold the Lever down and maintain opening during stretching or isometric exercises.

To install the Hand-Aid:

- Hold the jaw mobilizer in your left hand, and loosen the Fine Adjustment Knob so that the Lever can be squeezed all the way down.
- Take the hand-aid in your right hand with the narrow groove on top.
- Squeeze the Lever down, and place the open side of the hand-aid over the lowest part of the Lever. [Fig. 10a]
- Rotate the hand-aid one-quarter turn clockwise, so that the handle groove is over the handle, and release the squeeze.
- To use, slide the hand-aid towards the mouth. [Fig. 10b]
- To release, push the hand-aid away.

How to use the Range of Motion Scale

[Fig. 11 & 12]

The range of motion scale is designed to make mandibular motion measurement quick and easy. An area to record your progress is printed on the reverse side of the scale and a progress log comes with your device to track your progress.

To measure the Maximal Incisal Opening

[Fig. 11]

- Open your mouth as widely as possible without discomfort.
- Place the range of motion scale so that the notch rests on the edge of a lower incisor (tooth).
- Rotate the range of motion scale until it contacts an upper incisor.
- Take the reading at the point of contact.
- Record the measurement on the reverse side of the range of motion scale or in the separate patient progress log.

To measure Lateral Movement [Fig. 12]

- Rest the range of motion scale against the lower incisors with the teeth in gentle occlusion (the upper and lower teeth close together).

- Align the arrow with an interproximal space (space between two upper teeth). [Fig. 12a]
- Move the mandible laterally (side-to-side), and read the measurement on the Lateral Scale opposite the new position of the space. [Fig. 12b & 12c]
- Record your progress on the reverse side of the range of motion scale or in the separate patient progress log.

Getting started [Fig. 13]

Adjust the Fine Adjustment Knob so that the Lever moves freely. Let the Lever come all the way up.

- Hold the jaw mobilizer in the hand most comfortable for you.
- Relax and open your mouth.
- Place the mouthpieces between your upper and lower teeth, inside your lips. [Fig. 13a]
- Your front teeth should just touch the inside rim of the Upper Mouthpiece.
- Be careful to hold the jaw mobilizer level. Gently squeeze the Lever to push the Lower Mouthpiece down against your lower teeth. [Fig. 13b]
- Release at any time by relaxing your hand. [Fig. 13c]

Always begin and end each session by recording your beginning and ending measurements in the patient progress log. This will give you an accurate assessment of your progress, and will guide your clinician in directing your treatment.

Care and cleaning

Use soap/detergent and water to clean the TheraBite system. Rinse well, shake off excess water, and dry.

The range of motion scale is a disposable product, which should not be cleaned.

After cleaning the device, examine the bite pads on the Upper and Lower Mouthpiece and replace them if necessary. Replace bite pads only after ensuring that the mouthpieces are dry.

Caution: The mouthpieces are made of a material that tends to retain moisture. Therefore, do not attempt to use the jaw mobilizer until the mouthpieces are dry of all moisture. Do not soak the device in any liquid. Damage to the unit may result if force is applied to mouthpieces that are wet.

Caution: Avoid using a dishwasher to clean the TheraBite system. Prolonged cleaning with a dishwasher may weaken the bonding of the mouthpieces or cause discoloring.

Caution: Do not steam autoclave or gas autoclave the TheraBite system. Sterilization processes may weaken the bonding of the mouthpieces and cause failure under load.

Daily use

It is very important that the rehabilitation program recommended by your clinician is followed on a regular basis. To help you maintain your rehabilitation program, whether at work, while traveling, or at play, a Carrying Bag is included with the TheraBite system. The Carrying Bag is designed to keep all the components and accessories of the TheraBite system together. The Carrying Bag also has room for tissues and other convenient items.

Be sure to take the TheraBite system with you wherever you go and continue your rehabilitation program at regular intervals during both work and recreation, as your clinician has advised. Do not miss a session! Always measure and record the opening of your mouth before and after each session, using the range of motion scales and the patient progress log.

Rehabilitation programs for the TheraBite system

All users should receive adequate advice and instruction, as well as a personalized rehabilitation program, from their clinicians prior to using the TheraBite system. All rehabilitation programs should include regular monitoring, review, and evaluation by a properly trained clinician.

A commonly used exercise regimen is 7-7-7: Seven times per day. Seven stretches each time. Each stretch held for seven seconds. Individuals who suffer from muscle and/or joint pain may also benefit from longer stretch exercises in their rehabilitation programs.

An exercise regimen with longer stretches is 5-5-30: Five times per day. Five stretches each time. Each stretch held for 30 seconds.

After having used the TheraBite system for several months, you may be able to reduce your exercise regimen. Consult your clinician before making any changes to your rehabilitation program and/or exercise regimen.

Rehabilitation program for wider jaw opening

Stretching program

- Place properly cushioned mouthpieces between your teeth.
- Following the instructions of your clinician, slowly squeeze the Lever to open the mouthpieces. (Do not bite on the mouthpieces!)
- Allow your jaw to follow along with the motion of the mouthpieces. (Be careful not to tilt the jaw mobilizer up or down as you squeeze. Tilting may cause your jaw to move incorrectly.)
- Hold open as long as instructed by your clinician.
- Relax the squeeze, allowing your mouth to slowly close.
- Rest and repeat as instructed. If you find it difficult to hold the mouthpieces open, you may wish to use the hand-aid.
- A very slow opening motion may help to reduce pain or discomfort and may help to avoid muscle spasm.

For slow opening:

- Turn the Fine Adjustment Knob all the way in (counter/anti-clockwise).
- Gently squeeze the Lever with one hand while turning the Fine Adjustment Knob outward (clockwise) with the other hand.
- Continue to open until you have reached the limit prescribed.
- Relax, rest, and repeat.

Rehabilitation program to gain strength

Resisted Movement

- Insert the Mouthpieces and squeeze until you reach the opening recommended by your clinician.
- Without releasing the Lever, bite against the mouthpieces.
- Slowly release the Lever, allowing your mouth to close against the hand pressure. Your clinician will advise you how hard to squeeze while closing.
- Rest briefly, then repeat.

Rehabilitation program to develop coordination

Reeducation

The anatomically correct motion of the jaw mobilizer helps to train your jaw to move naturally.

- Insert the mouthpieces and bite very gently.
- Open your mouth while squeezing the Lever just hard enough to keep your teeth lightly touching the bite pads.
- After opening as wide as you comfortably can, release the Lever slowly and close, still keeping your teeth in contact with the bite pads.
- Repeat as often as instructed by your clinician so that your jaw muscles are trained to move correctly.

Troubleshooting

A build-up develops on the mouthpieces of the Jaw Mobilizer

Possible cause 1: Build-up may be due to dirt that has accumulated on mouthpieces after long periods of use.

Possible cause 2: Build-up may be due to the presence of calcium or other minerals in tap water.

What to do:

- Clean the mouthpieces with a non-glycerin alcohol.
- Use an emery board (nail file) to lightly file the surface of the mouthpieces.
- Rinse off the residue completely.
- Dry the mouthpieces and apply new bite pads.

Bite Pads do not stick properly to mouthpieces

Possible cause 1: bite pads may not stick properly if the mouthpieces contain moisture.

Possible cause 2: The bite pads were not properly placed on the mouthpieces.

What to do:

- Always make sure the mouthpieces are dry and free of build-up before applying the bite pads.
- Heat will make the adhesive on the bite pads tackier. Warm the mouthpieces under low heat with a hair dryer to evaporate moisture. Allow the mouthpieces to cool down to room temperature.
- Review the User Manual regarding bite pad application. Apply sufficient pressure to ensure good initial adhesion to the mouthpieces.

Discoloration occurs on the Jaw Mobilizer's surface and mouthpiece

Possible cause 1: Discoloration may occur due to the presence of iron in tap water.

Possible cause 2: Discoloration may also occur when the jaw mobilizer is exposed to direct sunlight, excessive moisture, or extreme temperatures.

In most cases, discoloration is purely cosmetic, and will not decrease the effectiveness of the device.

What to do:

- Do not soak the device in any liquid. Always dry the jaw mobilizer well after use and cleaning.
- Clean the mouthpieces with a non-glycerin alcohol.
- Keep the jaw mobilizer away from direct sunlight, excessive moisture, and/or extreme temperatures.
- When not in use, store the jaw mobilizer inside the Carrying Bag in a cool dry place.

Read the User Manual carefully to enhance your understanding of the appropriate use and care of the TheraBite system. Additional User Assistance Information is set forth in Part V, below.

PART II: INSTRUCTIONS FOR THE CLINICIAN

Setting Maximum Opening of the Jaw Mobilizer [Fig. 14-15]

Maximum openings of approximately 25 to 45 millimeters can be set by the position of the Range Setting Arm. [Fig. 14]

- Hold the device on its side and apply very gentle pressure to the Lever – not enough to move the Lower Mouthpiece.
- Use a pointed object (such as a pen or pencil) to push out the pin that connects the Range Setting Arm to the Lever. [Fig. 15a & 15b]
- Align the Range Setting Arm in the desired position and replace the pin. (Range change is approximately 5 mm per interval.)

The TheraBite system can be used without adjusting these settings, depending on the patient's ability to understand and implement instructions for the application of appropriate pressure.

The scale on the jaw mobilizer is for patient reference only. Use the range of motion scale for more accurate measurement.

PART III: ADDITIONAL INFORMATION

Limited Warranty

Atos Medical ("Manufacturer") warrants that the TheraBite Jaw Motion Rehabilitation System meets the Manufacturer's specifications and is free from manufacturing defects at the time of delivery.

This warranty is contingent upon proper use of the TheraBite Jaw Motion Rehabilitation System in the application for which it was intended and shall not extend or apply to any damage or defect resulting from product misuse, abuse, neglect, modification, alteration, unusual stress, or improper storage or handling.

Except for the express limited warranty set forth above, the manufacturer grants no warranties with respect to the TheraBite Jaw Motion Rehabilitation System, express or implied, either in fact or by operation of law, by statute or otherwise, and the manufacturer specifically and expressly disclaims any implied warranty of quality, warranty of merchantability, warranty of non-infringement of third-party rights.

The manufacturer's liability under the above warranty shall be limited to a refund of the distributor's purchase price. In no event shall the manufacturer be liable for the cost of procurement of substitute goods or for any special, consequential, or incidental damage for breach of warranty.

Date of Printing

See version number on the back cover.

Trademark

TheraBite is a registered trademark owned by Atos Medical AB, Sweden.

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©Atos Medical AB, 201108A, REF 90760-1