




Re-Order #	Description	Size	Unit
911548	Non-Folding Mat	1.5" x 26" x 68"	Each
911549	Non-Folding Mat	1.5" x 36" x 68"	Each

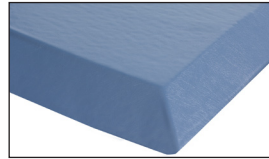

Beveled Edge



Purpose

Placed alongside a bed, the Roll-On Bedside Mat reduces the impact of a fall from bed. The tapered edges are designed for easy wheelchair roll-on and are safer for residents who might stumble on standard floor mats.

Easy to Use

Place Roll-On Bedside Mat on the floor next to the bed. Two mats will be required if side rails on both sides of the bed are lowered. **Important: Place the Roll-On Bedside Mat on the floor with the label reading “This Side Up facing up”.**

Important Notice

1. It is the responsibility of each facility’s appropriate medical professionals to determine if the Roll-On Bedside Mat is suitable as an injury-prevention device for a specific patient.
2. To prevent staff members from stumbling, make certain that all staff members, especially those working at night, are aware that the Roll-On Bedside Mat has been placed alongside a bed.

Easy Care

The Roll-On fall mat could be wiped clean with a spray and wipe cleanser. If disinfected, avoid using chlorine (bleach) chemicals.

Inspection

Inspect the cover before use. Do not use if the cover has cuts, rips cracks, or tears.

Turn product insert over for more of SkiL-Care’s Fall Control products.