


Stop-Strip

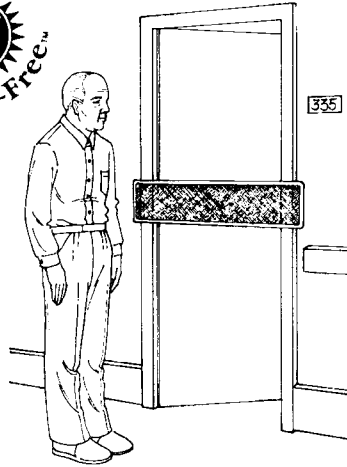

Reorder #	Color	Unit
<input type="checkbox"/> 909220	Orange	Each
<input type="checkbox"/> 909221	Yellow	Each



Purpose

Helps to prevent accidents by directing residents away from potential hazard areas. Promotes resident privacy. Placed across a room doorway, this brightly colored strip often stops the cognitively impaired wanderer from entering another resident's room. It also acts as a reminder for the resident inside a room not leave without first notifying a staff member. Secured with Velcro[®], the Stop-Strip is not a physical barrier, so it may be used as part of a restraint reduction program.

Important Warning

Do not place the Stop-Strip across fire and emergency exit doors.

Application

Peel the backing off the two Velcro[®] pressure-sensitive strips and attach them to the left and right sides of the door frame or to the wall on either side of the door frame. These strips should be positioned at midheight on the door frame or on the wall next to the door frame. (See illustration at right.) **IMPORTANT:** Attach the pressure sensitive strips to the corridor side of the door frame or wall so that the Stop-Strip is outside the room. (See illustration above.)

