Rev. 2 - 08/11_103652

Bauerfeind AG
Triebeser Strasse 16
07937 Zeulenroda-Triebes
Germany
Phone +49 (0) 36628-66-40 00
Fax +49 (0) 36628-66-44 99
E-mail info@bauerfeind.com

ViscoHeel®

Viscoelastische Einlagen
Viscoelastic heel cushions
Talonnettes visco-élastiques
Taloneras viscoelásticas

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Stand der Information: September 2009

EN Dear client,

thank you for choosing a product from Bauerfeind. ViscoHeel® is a high-quality medical product.

Please read through these instructions for use carefully and if you have any questions contact your medical provider, medical retailer or our Technical Service department.

ViscoHeel® are viscoelastic heel pads that provide relief for joints, ligaments, muscles and tendons (e.g. the Achilles tendon). ViscoHeel® pads are non-slip and can be applied in all types of walking, sports or work shoes.

⚠ ViscoHeel® must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.

Indications

ViscoHeel® N:

- Wear (arthritis) in the leg joints and relief for those with artificial joints (prostheses)
- Achilles tendon symptoms (achillodynia)
- Leg length discrepancies of up to approx. 1 cm (compensation)

ViscoHeel® K:

- Bow-legged or knock-kneed heel position (varus or valgus alignment)

Side effects

Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.

Fitting ViscoHeel®

ViscoHeel® N:

- Insert the ViscoHeel® pad into the shoe so that it fits precisely in the shoe's heel section.

ViscoHeel® K:

- The ViscoHeel® K pads are used for correcting a bow-legged or knock-kneed heel position. The heel pads are marked with the letter A and B.
- For a bow-legged heel position, insert the heel pad as follows: heel pad A in the right shoe, heel pad B in the left shoe. The heel pad should fit precisely in the shoe's heel section.
- For a knock-kneed heel position, insert the heel pad as follows: heel pad A in the left shoe, heel pad B in the right shoe. The heel pad should fit precisely in the shoe's heel section.

⚠ Important Information

- ViscoHeel® is a prescribed product that should be applied under a physician's guidance. ViscoHeel® should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product is intended to be fitted only once and to one patient.
- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- The use of ViscoHeel® during sport should be discussed with the medical provider.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.

Product care and washing instructions

Wash ViscoHeel® with a mild soap or detergent.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

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Bauerfeind Middle East FZ-LLC
Bauerfeind USA, Inc.