FlexiCol®

Hydrocolloid dressing
Apósito hydrocoloidal
Pansement hydrocolloïde

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Distributed in the USA by:

HARTMANN USA, Inc.
ROCK HILL, SC 29730, USA
1-800-243-2294
www.hartmanninfo.com

Products are covered under one or more of the following US patents:
6,309,500; 6,143,798; 5,998,694; 5,827,213; 5,704,905; 5,591,447

Instructions for use

FlexiCol is a self-adhesive absorbent hydrocolloid dressing with a gas-permeable polyurethane cover which is impermeable to liquids and bacteria.

Properties and mode of action

As the hydrocolloid absorbs the wound exudate, a gel is formed, ensuring a moist wound environment. The gel layer prevents FlexiCol from sticking to the wound. Dressing changes do not irritate granulation tissue or epithelial tissue and causes minimal discomfort for the patient.

FlexiCol is adhesive. The bevelled hydrocolloid edges ensure good adhesion even in problem areas. The surface of FlexiCol is dirt-repellent and water-repellent, facilitating daily personal hygiene.

Indications

FlexiCol is indicated for the management of wounds draining light to moderate amounts of exudate.

FlexiCol is not indicated for the management of 3rd degree burns or for wounds that extend into muscle, tendon or bone.

Mode of application

Select the correct size and shape of FlexiCol. The dressing should extend at least a half-inch beyond the edge of the wound on all sides to ensure that it stays firmly in place.

FlexiCol/FlexiCol Thin: remove the larger piece of the protective paper backing. Place FlexiCol on the wound holding the dressing by the portion still covered with a thin piece of protective paper backing. When the dressing is firmly in place, remove the remaining thin piece of protective backing paper and press all the edges of the dressing.

FlexiCol Sacral: remove the protective paper backing and press the dressing into the gluteal fold. The tip of the dressing can point upwards or downwards depending on the position and depth of the ulcer. Press the edges firmly to adhere the dressing.

FlexiCol Concave for the heel and elbow: remove the largest part of the protective paper backing, place the middle part of the dressing over the wound, remove the rest of the protective paper backing, then overlap the wings to fit.

FlexiCol should be changed when there is a visible discoloration of the dressing and a blister about the size of the wound forms in the center of the dressing. To change the dressing, lift FlexiCol at the edges and remove the whole dressing carefully. A layer of hydrocolloid gel may remain on the wound which should not be confused with pus. It will not impair the healing of the wound and can be rinsed off if desired.

In uncomplicated wounds, FlexiCol can be left in place for intervals of up to 7 days or until the wound has completely healed.

Special precautions

FlexiCol can only be effective when a holistic assessment of the patient and the wound has taken place. Treatment with FlexiCol cannot replace treatment of the cause of the impaired healing.

General instructions

Store between 41 °F to 86 °F (5 °C to 30 °C). Contents sterile unless peel packs are damaged. Do not re-sterilize. Keep out of reach of children.

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