



THE COMFORT COMPANY

Quietly making lives more comfortable™

LATERAL SUPPORT ASSEMBLY



1 Remove harness and lateral pads from package.

2 Separate lateral pads from harness.

3 Attach harness to wheelchair backrest.
a) Wrap the single strap around the top/bottom of the wheelchair backrest. Secure using the hook/loop.
i. Note: There is an additional strap included in the package for tall wheelchair backrests (i.e. reclining wheelchairs).
b) Wrap the horizontal straps around the outside of the wheelchair. Secure using the hook/loop straps.

4 Re-attach the lateral pads to the harness in the desired location.