



## Manipulation Board

**Item Sku:** 5312/081029784

**Item Description:**

Hardware and closures typically used in the home are mounted on a 23" x 14-3/4" wooden board. Eight devices mounted on a hardware base include a door chain lock, electric outlet plate, padlock, key lock, faucet, hook & eye lock and luggage lock. Pre-drilled for mounting. Latex free

**Indications:**

This Manipulation Board can be used as an evaluation tool as well as therapeutic exercise to increase your clients' ability to perform day-to-day routines and independent living. Helps clients exercise fine motor skills and gains strength in performing common tasks. Hardware and closures typically used in the home are mounted on the wooden board. Help Alzheimer's clients by occupying them with familiar tasks.

**Care and Cleaning:**

Manipulation Board can be wiped clean with a mild detergent – no bleach, no heavy duty cleaning products - to clean. Do not submerge in water. Dry with soft cloth.

**Procedure:**

- Use manipulation board on its own or place vertically in the optional wooden base
- Use the board to facilitate adapted task performance by individuals with injuries or disease that affect hand function

Accessory Parts available:

Item Description: Optional Base

Item Number: 531201



**Distributed by:**

**Patterson Medical**  
28100 Torch Parkway Ste. 700  
Warrenville, IL 60555, USA  
Phone: 800-323-5547  
Direct Phone: 630-393-6000  
Fax: 800-547-4333  
Direct Fax: 630-393-7600

**Patterson Medical**  
W68 N158 Evergreen Blvd  
Cedarburg, WI 53012, USA  
Phone: 800-323-5547  
Fax: 800-547-4333

**Patterson Medical Canada**  
6675 Mill Creek Drive, Unit 3  
Mississauga, ON L5N 5M4  
Phone: 800-665-9200  
Direct Phone: 905-858-6000  
Direct Fax: 905-858-6001

**Authorized European Representative  
Patterson Medical, LTD.**  
Nunn Brook Road  
Huthwaite, Sutton-in-Ashfield  
Nottinghamshire, NG17 2HU, UK  
**International**  
Phone: +44 (0) 1623 448 706  
Fax: +44 (0) 1623 448 784