Instructions for use

Product description

Sorbalgon is a non-woven wound dressing (or ribbon) consisting of calcium alginate fibres and polysorbate. In contact with sodium salts – as found in wound exudate – the alginate fibres become gelatinous creating a moist wound healing environment.

Properties and mode of action

The non-woven structure of the dressing combined with its physical and chemical properties result in a high absorbency – particularly important in the treatment of moderate to heavily exuding wounds. The material adapts remarkably to the contours of the wound preventing desiccation. For the treatment of cavity wounds, Sorbalgon ribbons are available.

Indications

For all moderate to heavily exuding wounds.

Mode of application

Cleanse the wound in accordance with standard facility practice. Apply the dressing to the wound or plug the wound with a Sorbalgon ribbon dressing and secure with a suitable secondary dressing such as Omnifix. Sorbalgon may be left on the wound for extended periods (up to several days), depending on the level of wound exudation. The dressing should be changed – once it has fully turned into a gel – in accordance with nursing standards. Should any fibres remain in the wound, these may be easily removed by a wound irrigation solution and are broken down naturally by the body.

Side effects

None have been observed or reported.

Contraindications

Do not use Sorbalgon/-T on any patients who may be allergic to any of its ingredients.

Special precautions

Medical assessment on the condition of the wound and the causes of the delayed wound healing are necessary before treating wounds with impaired healing. Treatment with Sorbalgon cannot replace treatment of the cause of the impaired healing. In the event of full thickness second degree and third degree burns, Sorbalgon may be used under medical supervision following debridement of necrotic tissue or surgical treatment.

General instructions

Store below 40 °C in a dry place. Contents sterile unless peel packs are damaged. Do not resterilize. Keep out of reach of children.

Presentations

2 in. x 2 in. (5 cm x 5 cm)	8 boxes per case
4 in. x 4 in. (10 cm x 10 cm)	7 boxes per case
4 in. x 8 in. (10 cm x 20 cm)	13 boxes per case
Sorbalgon ribbon for packing wounds 12 in. (30 cm)	10 boxes per case

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HARTMANN USA, Inc., Rock Hill, SC 29730
PAUL HARTMANN AG • 89522 Heidenheim, Germany