## DANSK

### STØRRELSE

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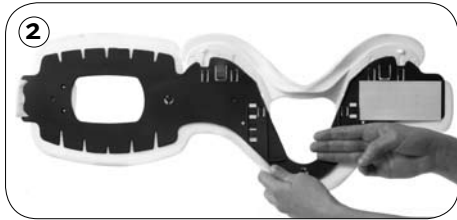
## ENGLISH

### SIZE

Position your lowest finger on the sizing line. Select the size closest to the height of your fingers (Figure 1). Position the collar with the selected size (Figure 2). Press the lock into the collar until secured. Verify by attempting to readjust (Figure 3).

### APPLICATION

#### Supine Position

Maintain neutral alignment. Select appropriate size. Slide collar under patient's neck (Figure 4).

#### Seated Position

Maintain neutral alignment. Select appropriate size. Slide collar behind patient's neck. Position chin support.

#### Secure Collar

Position chin support. Tighten collar and attach Velcro® strap (Figure 5).

### PRECAUTIONS

Do not rely solely on extrication collar for sufficient immobilization. Do not use improperly sized collars. Always follow local EMS authority directions.

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**EN - Caution:** This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

Patriot® Collar is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.

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© Copyright Ossur 2008