



# Comperm Sizing Guide

## Comperm® LF and Comperm®-PD LF

Tubular bandages that maximize support and minimize constriction.

Part #	Description	Sizing Recommendation	Diameter
83010000	Comperm LF Size A	Very Small Limbs or Feet	1.63"
83020000	Comperm LF Size B	Small Limbs	2.5"
83030000	Comperm LF Size C	Hands, Feet or Small Lower Legs	2.75"
83040000	Comperm LF Size D	Lower Legs or Arms	3"
83050000	Comperm LF Size E	Knees or Small Thighs	3.5"
83060000	Comperm LF Size F	Large Knees or Thighs	4"
83070000	Comperm LF Size G	Large Thighs	5"
83080000	Comperm LF Size J	Small Trunks	7"
83090000	Comperm LF Size K	Medium Trunks	8.5"
83100000	Comperm LF Size L	Large Trunks	13"
83120000	Comperm - PD LF Small	Large Arms and Small Legs	3"
83130000	Comperm - PD LF Large	Large Knees and Calves	4"

The above guidelines are suggested usages. Each patient will vary in size and shape and their optimal size will vary. Comperm padding bandages (Comperm - PD) are for use with patients who need or would like extra cushioning to protect the limb.

### Comperm Compression Guidelines using mmHg:

For the best fit, measure the widest circumference of the extremity.

	Low Compression	Medium Compression	High Compression
<b>Circumference</b>	<b>5 - 10 mmHg</b>	<b>10 - 20 mmHg</b>	<b>20 - 30 mmHg</b>
6" - 10" (15 - 25cm)	Sizes D or C	Sizes C or B	Sizes B or A
10" - 14" (25 - 35cm)	Sizes G or F	Sizes F or E	Sizes E or D
14" - 18" (35 - 45cm)	Size J	Sizes G or F	Sizes F or E
18" - 20" (45 - 50cm)	Sizes J or H	Size G	Size F
20" - 24" (50 - 60cm)	Size K	Size J	Size G



The above guidelines are for using Comperm in a single layer (not doubled-up) and are suggested usages based on test results for mmHg at different circumference ranges. Each patient will vary in size and shape therefore their optimal size will vary.