
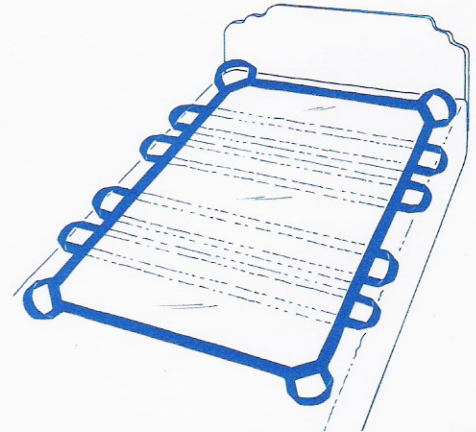


SUPER-SLINGS

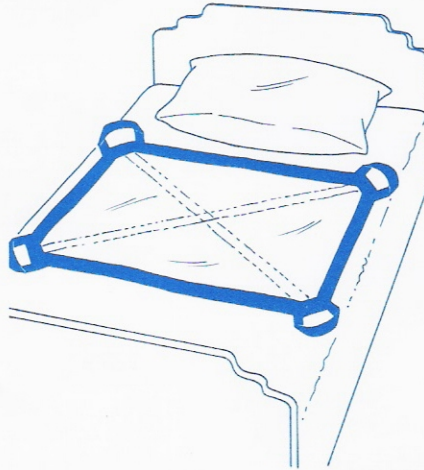

Reorder #	Size	Description	Weight Capacity
<input type="checkbox"/> 253010	46" x 46"	8 -Handle (For use by four caregivers)	500 lbs.
<input type="checkbox"/> 253020	46" x 46"	4 - Handle (For use by two caregivers)	250 lbs.
<input type="checkbox"/> 253030	36" x 72"	12 - Handle (For use by six caregivers)	500 lbs.

Purpose

Super-Slings are designed to facilitate patient transfer, lifting, and positioning. They may also be used for emergency evacuations when wheelchairs or gurneys are either unavailable or not practical. Super-Slings reduce exposure to back injury by permitting multiple caregivers to lift, move, transfer and position patients. Skil-Care's Super-Slings are made from low-friction nylon that enables caregivers to slide patients easily on all surfaces, including beds and gurneys.

Important Notices

- Determine how many caregivers will be required to lift, carry, or move a specific patient.
- Be certain to follow your facility's established policies for lifting, carrying, moving, or positioning patients.
- Make certain that the patient's head is supported (either by positioning the Super-Sling beneath the head or by having a caregiver support the head with his/her hands) when lifting or carrying a patient.

Easy Care

All Super-Slings may be washed and dried at temperatures that do not exceed 180°F (180°C). Do not use bleach.