

I N S T R U C T I O N S

Extra Wide Tall-Ette® Toilet Seat with Arms and Legs

Item 725881000

Instructions:

1. Insert the two safety bars into the four holes in the top of the plastic seat.
2. Push the four rubber feet onto the ends of the safety bars.

(Note: Dipping the rubber feet in soapy water will make them slide on easier.)

3. Turn the seat over and place the Lok-In-EI® Bracket on the seat by pressing the two large pins into the two wells provided. Turn seat right side up. The flat bar on the Lok-In-EI® Bracket will extend from the rear of the seat.
4. Lift the toilet bowl seat to the raised position.
5. Slip the flat bar of the Lok-In-EI® Bracket under the hinge bar of the toilet seat. The bottom ring of the elevated seat must fit inside the toilet bowl.
6. Adjust the legs so the seat sits firmly on the bowl and the legs rest on the floor.



Note: Once installed, if you find that the legs do not reach the ground simply cut some of the foam tubing from the arms. This will allow the seat to slide further up the legs. See instructions on the back.

7. The seat is now ready for use. It may be cleaned by using a mild detergent solution and sterilized with alcohol.

Caution: Do not use this product without the Lok-In-EI® Bracket. Do not use this product if the seat does not fit inside the toilet bowl rim.



A Division of SP Industries

661 Route 23 South • Wayne, NJ 07470

973-628-7600 • email custservice@maddak.com • www.spableware.com

I N S T R U C T I O N S

Cutting the Foam Tubing:

If the legs do not reach the ground after installing, simply cut some of the foam tubing from the arms. This will allow the seat to slide further up the legs.

To do this:

1. Measure the distance between the floor and the legs. Whatever the distance is, you will need to cut that much off of the bottom of the foam tubing on each arm. So, if the distance between the floor and the legs is 1" (2.54 cm), you will need to cut off 1" (2.54 cm) of the tubing on each arm.

Do not cut tubing on the top of the armrests.

After measuring, cut the foam tubing from the bottom of the arms.



2. With a ruler, measure 1" (2.54 cm) from the top of the elevated toilet seat up the arm and mark the tubing. Do this for all four legs.
3. Cut the tubing using a sharp knife or box cutter.

Note: Always cut tubing from the part closest to the top of the elevated toilet seat. Do not cut from the top of the armrests.



A Division of SP Industries

661 Route 23 South • Wayne, NJ 07470

973-628-7600 • email custservice@maddak.com • www.spableware.com