

Gel Armrest Pads 562054 & 562055

Installation Instructions:

Check alignment of holes in handle of wheelchair to those on the gel pad-
If holes match up, use 2 each 10-32 x 1-1/4" screws (not included) to attach gel
pad to the bar- tighten screws.

If holes do not match, mark the location of the gel pad holes in the desired
location on the armrest bar and drill holes in the tube slightly larger than the
screws. Use 2 each 10-32 x 1-1/4" screws (not included) to attach gel pad to
the bar- tighten screws.

Cleaning Instructions:

The exterior surface of the gel pad can be washed with any commercial
detergent by scrubbing and rinsing with a wet sponge or cloth. Do not
submerge gel pad in water or put in washing machine as gel may dissolve.



| |
|---|
| Call toll free: 1-800-323-5547 In Canada 1-800-665-9200 Fax toll free: 1-800-547-4333 In Canada 1-905-566-9207 |
|---|