

Gel Calf Support Pad 562058

Installation Instructions:

The gel calf support has 3 hook and loop straps on each side. Attach the 2 parallel straps around the leg rest extension tubes and fasten together behind pad to adjust desired depth of pad. The third strap runs out of the top of the calf support at a 45 degree angle and is wrapped around the upper section of the leg rest to maintain the proper height adjustment of the pad.

Cleaning Instructions:

The exterior surface of the gel pad can be washed with any commercial detergent by scrubbing and rinsing with a wet sponge or cloth. Do not submerge gel pad in water or put in washing machine as gel may dissolve.



Call toll free: 1-800-323-5547 In Canada 1-800-665-9200 Fax toll free: 1-800-547-4333 In Canada 1-905-566-9207
