

## Toileting Aid

081165299 (6014)—12" Long

081169416 (6219)—15" Long  
081536242—15" Long (Pack of 10)

081551621—18" Long

### INSTRUCTIONS FOR USE



1. Select a length of toilet tissue which should be approximately 6 - 10 sheets long. Fold the length of tissue in half.



2. Grip one end of the toilet tissue in the toileting aid, then wrap the opposite end of the tissue paper around the end of the aid, covering the vinyl end. Tuck the loose end into the wrapped tissue.



3. Make sure you keep a closed grip on the handles to hold the toilet tissue in place for cleaning.



4. It is recommended to be used between the legs, from the front or from behind, while seated on the toilet.



5. To remove soiled material, open the grip on the handles and dispose into the toilet. If necessary wrap the soiled material with a length of clean tissue and remove with your hand. Repeat the above process as required.

To clean the vinyl end of the toileting aid, simply use a moist wipe or a disposable cloth with mild soapy water.



#### Distributed by: Performance Health Supply Inc.

28100 Torch Parkway, Ste. 700  
Warrenville, IL 60555, USA  
Phone: 800-323-5547  
Fax: 800-547-4333

#### Manufactured by: Performance Health Supply Inc.

W68 N158 Evergreen Blvd.  
Cedarburg, WI 53012, USA  
Phone: 800-228-3693

#### Distributed by: Performance Health Canada, Inc.

6675 Millcreek Drive, Unit 3  
Mississauga, ON L5N 5M4  
Phone: 800-665-9200  
Fax: 877-566-8611

#### Authorized European Representative Performance Health International, LTD.

Nunn Brook Road  
Huthwaite, Sutton-in-Ashfield  
Nottinghamshire  
NG17 2HU, UK  
International  
Phone: +44 (0) 1623 448 706  
Fax: +44 (0) 1623 448 784