

## Finger Spring Low Tension Flexion

Single Patient Use Only • Rx Only

### IMPORTANT INFORMATION

The following information should be read prior to use of this device.

### INTENDED USE

This splint fits from the head of the proximal phalanx to the distal interphalangeal (DIP) joint. This splint provides a light tension at the proximal interphalangeal (PIP) joint, to position the PIP joint in a flexed position. The distal phalanx is free to allow functional use of the finger pad.



To determine splint length, measure involved finger as indicated in the diagram.

AA	1 1/4"	3.2 cm
A	1 1/2"	3.8 cm
B	1 3/4"	4.4 cm
C	2 1/4"	5.7 cm
D	2 3/4"	7.0 cm

### DIRECTIONS FOR USE

#### Fitting the Spring

1. Slide splint onto finger so the proximal pad lies over the dorsum of the phalanx, proximal to the joint, and the

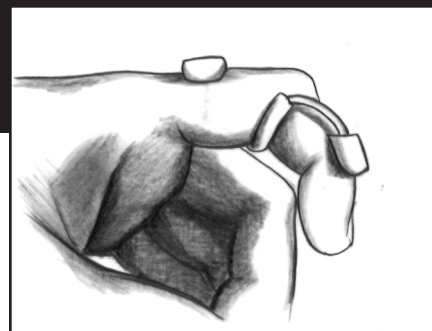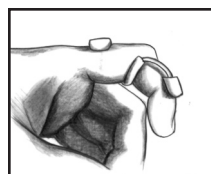

distal dorsal pad lies over the dorsum of the phalanx distal to the joint.

2. Three points of pressure are utilized to position the finger in flexion.

#### Adjusting the Spring

1. Slight adjustments in the degree of flexion can be made.
2. Bend lateral wires, equally, as indicated in the illustration.
3. Avoid rotating the pads of the splint.



### CARE INSTRUCTIONS

Splint can be cleaned by hand washing with a non-abrasive detergent. Air dry before wearing.



**Manufacturer**  
DeRoyal  
200 DeBusk Lane  
Powell, TN 37849 U.S.A.  
888.938.7828  
865.938.7828

Part #1-601  
©2009 DeRoyal. All Rights Reserved.  
Revised 1/09



## Finger Spring Low Tension Flexion

Single Patient Use Only • Rx Only

### IMPORTANT INFORMATION

The following information should be read prior to use of this device.

### INTENDED USE

This splint fits from the head of the proximal phalanx to the distal interphalangeal (DIP) joint. This splint provides a light tension at the proximal interphalangeal (PIP) joint, to position the PIP joint in a flexed position. The distal phalanx is free to allow functional use of the finger pad.



To determine splint length, measure involved finger as indicated in the diagram.

AA	1 1/4"	3.2 cm
A	1 1/2"	3.8 cm
B	1 3/4"	4.4 cm
C	2 1/4"	5.7 cm
D	2 3/4"	7.0 cm

### DIRECTIONS FOR USE

#### Fitting the Spring

1. Slide splint onto finger so the proximal pad lies over the dorsum of the phalanx, proximal to the joint, and the

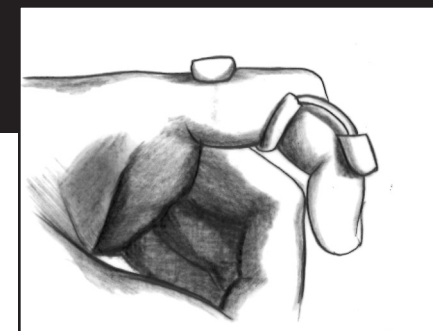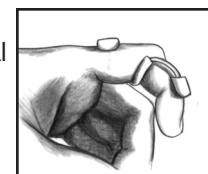


distal dorsal pad lies over the dorsum of the phalanx distal to the joint.

2. Three points of pressure are utilized to position the finger in flexion.

#### Adjusting the Spring

1. Slight adjustments in the degree of flexion can be made.
2. Bend lateral wires, equally, as indicated in the illustration.
3. Avoid rotating the pads of the splint.

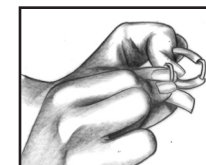

### CARE INSTRUCTIONS

Splint can be cleaned by hand washing with a non-abrasive detergent. Air dry before wearing.



**Manufacturer**  
DeRoyal  
200 DeBusk Lane  
Powell, TN 37849 U.S.A.  
888.938.7828  
865.938.7828

Part #1-601  
©2009 DeRoyal. All Rights Reserved.  
Revised 1/09