Rolyan® Preformed Anti-Spasticity Ball Splints

Rolyan® Preformed Anti-Spasticity Ball Splint: A419-LS through RL; A419-19 through A419-24; 5505-24 through 5505-29; 7974-01-01 through 7974-03-02

Rolyan® Hand-Based Anti-Spasticity Ball Splint: A419-1 through A419-8

Rolyan® Anti-Spasticity Ball Splint with Slot & Loop Strapping: A419-300 through A419-307; 5505-16 through 5505-21

Rolyan® Dorsal Anti-Spasticity Ball Splint: A419-11 through A419-16

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints listed. Figures referred to in the instructions are on page 6.

INDICATIONS

- To help prevent deformity in the presence of mild to moderate hypertonicity
- May be used following CVA and traumatic brain injury, and with Parkinson's disease

Anti-Spasticity Ball Splint with Slot & Loop Strapping

- May be used with severe hypertonicity

CONTRAINDICATIONS

- Not in the presence of wrist or hand fractures
- Not for use with severe edema
- Not for use over open wounds and infected areas

PRECAUTIONS

- **These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away

from sources of heat such as ovens, hot water, open flames, and sunny car windows.

- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

INSTRUCTIONS FOR USE

Modification

These splints, which are made from ¼" (3.2mm) Polyform® splinting material, can be modified for a better fit. Heat the area to be modified using a heat gun or water that is heated to 150° to 160°F (65°–70°C) for one minute or until pliable.

- Heating the entire splint may cause it to lose its preformed shape.
- Polyform has a high degree of drape—for better results when modifying the material, shape it with stroking movements.
- Do not heat any of the splints near the rivet that holds the straps in place. Do not heat the **Anti-Spasticity Ball Splint with Slot & Loop Strapping** near the strapping or the plastic base of the slot-and-loop strapping.

Finger Strapping

Preformed Anti-Spasticity Ball Splint (see photo A)

Hand-Based Anti-Spasticity Ball Splint (B) Dorsal Anti-Spasticity Ball Splint (C)

- Strap fingers as shown.
- For patients with tight finger or wrist flexors, position the **Dorsal Anti-Spasticity Ball Splint** upside down and parallel to the forearm, with the hand portion of the splint next to the forearm. Slip the fingers and thumb through the cut-out area and use the forearm of the splint as a lever to place the splint onto the dorsal surface of the patient's forearm (D).

Anti-Spasticity Ball Splint with Slot & Loop Strapping (E)

- Put the splint into position by sliding the fingers under the elastic strapping. If necessary, adjust the strapping for proper fit—feed it through the slotted plastic base on the underside of the splint to increase or decrease the amount of elastic needed to secure each finger.
- Attach two self-adhesive hook pieces to the ulnar and radial borders and secure the ends of the strapping to them.

Forearm Strapping

Preformed Anti-Spasticity Ball Splint (A)

Dorsal Anti-Spasticity Ball Splint (C)

Anti-Spasticity Ball Splint with Slot & Loop Strapping (E)

- Apply the two long straps as appropriate.
- The **Preformed Anti-Spasticity Ball Splint** and the **Anti-Spasticity Ball Splint with Slot & Loop Strapping** include soft foam padding, which may be placed on the straps for increased comfort.

Wrist Strapping

Hand-Based Anti-Spasticity Ball Splint (B) Anti-Spasticity Ball Splint with Slot & Loop Strapping (E)

These splints include a padded Polyform wrist brace, which may be heated (see **Modification**) and molded to the hand. Place it on the dorsal surface of the wrist with the longer edge placed distally and the padded side toward the skin. Mold it to the wrist. Position the wrist stabilizing strap, threaded through the padded brace, around the wrist.

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Patterson Medical, Ltd.