

Rolyan Arm Restraint System

Item No

8022-01	Pediatric Small*	7" x 5" (18cm x 13cm)
8022-02	Pediatric Medium	9" x 7" (23cm x 18cm)
8022-03	Pediatric Large	10" x 8" (25cm x 20cm)
8022-04	Adult Medium	10" x 11" (25cm x 28cm)
8022-05	Adult Large	12" x 13" (30cm x 33cm)

*available with two straps only

Description

Soft, hook sensitive nylon foam laminated cover with flexible plastic insert

Indications

Used to help maintain arm in an extended position preventing elbow flexion keeping hands away from face, mouth, head or IV tubing.

CONTRAINDICATIONS

- Not for use when bone is present
- Not for use over open wounds and infected areas
- Not intended for home use

PRECAUTIONS

- Product should be fitted by a health care professional.
- Prior to donning product, the product should be checked for any obvious deterioration of cover as serious injury could result if plastic insert is exposed.

Instructions For Use:

1. Position the Rolyan Arm Restraint System with elbow centered, in the restraint, wider end towards upper arm. The end with hook straps should be towards the truck of the patient.
2. Wrap product around patient's arm with the straps attaching to hook sensitive nylon top of restraint.
3. Product should overlap by at least 1".
4. Do not apply product too tightly to ensure adequate circulation.
5. Check patient often.



Do Not Bleach



Do Not Iron



Latex free



Tumble Dry Normal Setting



Do Not Dry Clean



(<130°F/54°C) Machine wash; do not exceed 130°F/54°C



Attention: See Instruction For Use



Patterson Medical
1000 Remington Blvd.
Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310