
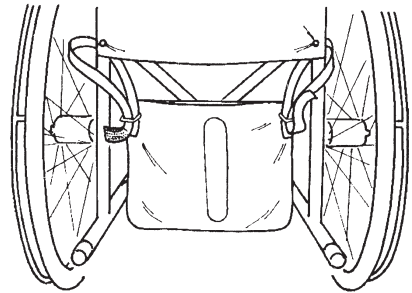


URINARY DRAINAGE BAG HOLDER

| Reorder # | Description | Unit |
|------------------|---------------------------------------|-------------|
| ☐102010 | Urinary Drainage Bag Holder | Each |
| ☐102015 | Thru-View Urinary Drainage Bag Holder | Each |

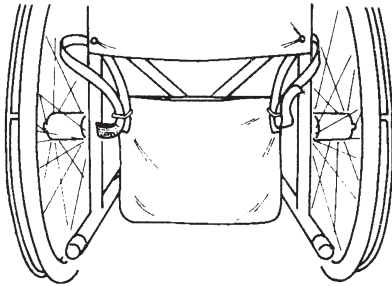

Purpose

Restores the dignity of your catheterized patients by concealing urinary drainage bags from public view. Protects drainage bags from damage, contains spills or leaks, and provides a convenient place to store excess tubing.

Easy to Use

Universally sized to accommodate all drainage bags, this holder hangs from wheelchairs, geri-chairs, or bed rails. Place straps around the chair frame or bed rail and through the plastic rings at both sides of the bag. (See illustrations above.) Adjust the bag to desired height. Secure by pressing the hook tabs against the loop material. Place the urinary drainage bag in holder. Excess tubing may be looped and stored in the holder.

Easy to Clean

Made from bacteriostatic vinyl, Skil-Care's Urinary Drainage Bag holder cleans easily with soap and water. Hand wash in warm water. Do not launder or place in drier.