





## Comforter Splint

For nighttime positioning of the wrist and fingers to reduce ulnar drift at the MP joints caused by rheumatoid arthritis, neurological impairment or trauma. May also be used for daytime use to rest and relieve pain.

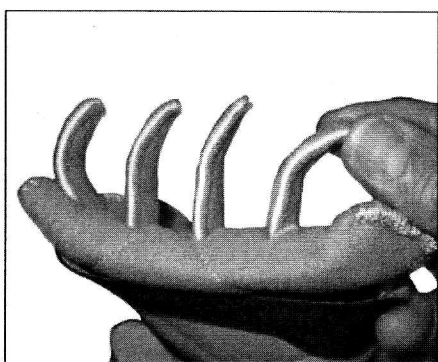


The bendable finger stays may be carefully straightened to ease putting the splint on and taking it off.

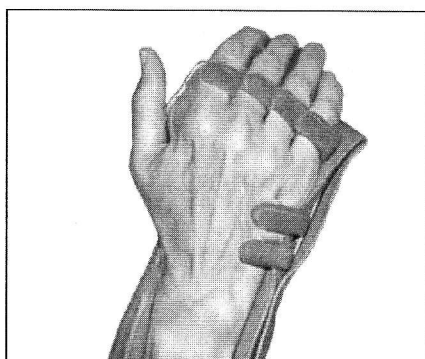

Slide hand into splint and gently shape each finger stay into a half circle over the finger. Avoid creating sharp curves in fingerstays. To prevent pressure in web spaces, stays may be rotated at their rivet attachments.



Apply straps. The finger holes can be used to pull the straps on without having to use a strong pinch to grasp the straps.

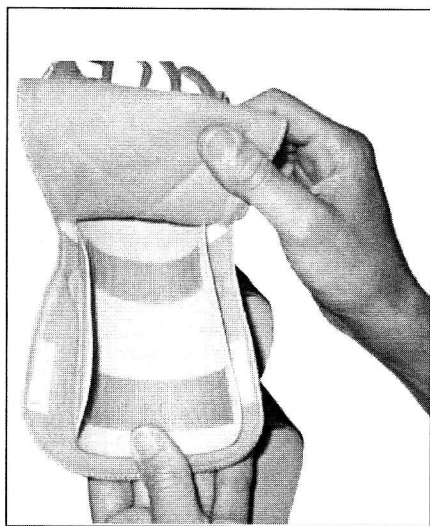

The Comforter cover is removable for easy cleaning. See back for care instructions.



Stevensville, MD 21666 U.S.A.  
Made in the U.S.A.

## ADJUSTMENTS

The base of the Comforter splint is made from 1/16" thick Kydex. Adjustments should be made by a qualified health care professional. **CAUTION:** Kydex is a high temperature plastic and can not be placed directly on the skin when hot and pliable.

Remove cover and hook straps as needed. To heat, hold heat gun 2 to 3 inches away for 30-60 seconds. Trim warm material with scissors and smooth edges with an emery file if needed. Avoid heating around the finger stays.

## TO ORDER

Measure the width across the 2nd to the 5th MP joints.

	Width at MP's	Right	Left
Small	2 3/4" - 3"	P3002-R2	P3002-L2
Medium	3" - 3 3/8"	P3002-R3	P3002-L3
Large	3 3/8" - 3 5/8"	P3002-R4	P3002-L4

Order Toll Free **888.378.7763** (U.S only)

Local/Int. 410.604.6393

24 hour Fax 410.604.6398

Monday - Friday 8:30 - 5:30 pm EST

## CARE INSTRUCTIONS



Remove cover and wash splint with soap and warm or cold water. The Comforter cover may be hand or machine washed and air dried. Do not place cover in dryer.

**Material:** Kydex plastic, Nylon cover, Neoprene straps.


©2005, 3-Point Products, Inc.

Authorized European Representative  
Emergo Europe  
Molenstraat 15  
2513 BH The Hague  
The Netherlands




[www.3pointproducts.com](http://www.3pointproducts.com)

For daytime use to correct moderate to severe ulnar drift, choose the Radial Hinged Ulnar Deviation Splint.

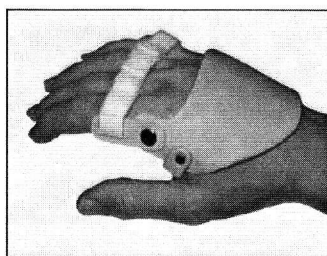

Radial Hinged Ulnar Deviation Splint

For daytime use to correct mild to moderate ulnar drift, choose the Polycentric Hinged Ulnar Deviation.

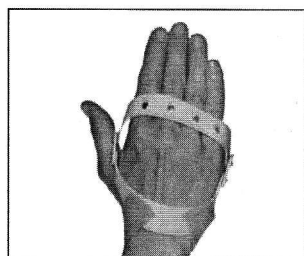

Polycentric Hinged Ulnar Deviation

3-Point Products is your source for products designed for comfort, fit and function.