**A**

**B**

## INDICATIONS

- To immobilize the knee and help reduce pain
- May be used following surgery, fractures, and sprains
- May be used for arthritis, luxation of the patella, Osgood-Schlatter syndrome, and osteochondritis of the knee

## CONTRAINDICATIONS

- Not for use over open wounds or infected areas
- Not in the presence of severe edema

## PRECAUTIONS

- **This immobilizer is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **The immobilizer should fit snugly and comfortably without interfering with normal circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

## CARE AND CLEANING

- The immobilizer will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car

windows.

- The immobilizer can be cleaned with soap and lukewarm water.
- The straps and fabric flaps can be wiped with warm water and mild soap. Allow to dry completely before reapplying the immobilizer.

## INSTRUCTIONS FOR USE

1. For patient comfort during molding, place stockinette on the area that will be covered by the immobilizer. If softening the immobilizer in a convection oven, use two layers of stockinette, with more over newly healed scars or skin grafts.

2. Heat the  $\frac{1}{8}$ " (3.2mm) **Aquaplast®-T** immobilizer as follows:

**Heat pan:** Heat in water at a temperature of 160° to 170°F (70°–75°C) for one minute or until transparent. Hold the flaps and straps out of the water. Remove from heat pan at an angle to let water drain off without stretching the immobilizer and place on a flat surface. Pat dry with a towel. Blot any water from the flaps and straps.

**Convection oven:** Heat without overlapping or folding on a baking sheet with a non-stick surface in an oven preheated to 195° to 205°F (90°–95°C) until transparent (softening may take up to 10 minutes). To prevent stretching and accidental bonding, two people should remove the item from the baking sheet. Place on a flat surface.

3. With the patient supine, the foot elevated, and the knee positioned appropriately, position a healthcare professional at each side of the leg.

4. Place the immobilizer under the leg. Bring the immobilizer, working from proximal to distal, up to the posterior leg surface. Center the knee in the opening and fasten the hook-and-loop closure on the flaps across the front of the leg (**A**). If necessary, gently pull the material evenly around the leg, from inside the immobilizer toward the flaps. Do not fasten the straps at this time.
5. Stroke the immobilizer to mold it snug against the leg.
6. Leave the immobilizer in place until it is completely cool. Continued stroking will help it maintain its shape.
7. Fasten the straps on the anterior leg surface as follows:
  - First, fasten the straps closest to the knee, the proximal one and then the distal one.
  - Next, fasten the strap at the proximal end of the immobilizer, followed by the one at the distal end.
  - Finally, fasten the remaining two straps.
8. The immobilizer can be trimmed and the edges flared as necessary. Areas that have been stitched cannot be trimmed.
9. For greater comfort, self-adhesive padding can be applied to the proximal and distal borders after the immobilizer is molded. Apply the padding first to the inner surface to minimize wrinkles between the immobilizer and the skin. Any wrinkles that form can be cut away with a scissors.
10. A properly applied immobilizer is shown in **B**.

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**Patterson Medical, Ltd.**