## Rolyan® Functional Position Splint:

A310-SFX, MFX, LFX, SPE, SE, ME, LE, SS1, SS2, MS1, MS2, LS1, LS2, SA1, MA1, LA1, 500 through 505, 600 through 607, A311-200 through 205, A923-103 through 106

A

B

C

*Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints listed.*

### INDICATIONS

- To help prevent deformity from hypertonicity
- To support inflamed joints or to immobilize joints after injury or surgery

### CONTRAINDICATIONS

- Not for the acute phase of most wrist or hand fractures

### PRECAUTIONS

- This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- Be careful that the straps are not so tight that they interfere with circulation.
- After heating splinting material, always check its temperature before applying it to the patient.

### Splints made from Rolyan AirThru\* material

- When opening bag after heating material, avoid contact with hot vapors from bag.

### CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water and allowed to dry thoroughly before reapplication.

### PROCEDURE

**Splints made from AirThru material:** These splint blanks fit the right or left hand. **Other splints:** The splint blank can be applied to either hand—simply invert the splint blank.

1. Heat splint blank as follows:

**Splint blanks made from AirThru material—**Soften these splints, which are packaged in a heat-sealed plastic bag, in heat pan of water at a temperature of 160° to 170°F (70°–75°C) for approximately 2 to 3 minutes. Cut open bag between heat seal and pressure-sensitive strip. If reheating is necessary, return splint blank to bag, close pressure-sensitive seal completely, and heat.

**All other splints—**Soften in water heated to approximately 160°F (70°C) until pliable.

- (Refer to the instructions for Rolyan splinting materials for information about working with specific splinting materials.) Remove from heat pan and place on a towel on a flat surface. Blot excess water.
2. Flare the web space by stretching the outer ridge of the splinting material so it contours to the web space (A).
  3. If possible, place patient's hand in supination to achieve a gravity-assisted position.
  4. Mold the splint blank on the hand and forearm as shown in B.
  5. The splint can be trimmed and the edges flared if necessary.
  6. Apply appropriate straps. A completed splint is shown in illustration C.

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